

International
Oxygen Concentrators
IRC5LXAW
IRC5LXAW Platinum 5
(Units Built After Serial No. 02G737514)

IRC5LX02AW
IRC5LX02AW Platinum 5
(Units Built After Serial No. 02F595196)



Parts Catalog Usage Guide

The information contained in this document is subject to change without notice. This catalog contains service parts for Form No. 99-176. If you have any questions regarding this catalog, please contact one of our Customer Service Representatives.

For Customer Service Support Call:

1-800-333-6900

For Faxed In Orders Call:

1-800-678-4682 or

1-440-366-0709

For Technical Service Call:

1-800-832-4707

Literature Can Be Found at:

www.invacare.com or

1-800-828-6272

SYMBOL DEFINITION

Throughout this catalog you will notice letters to draw your attention to specific footnotes which are located at the bottom of the page or below specified charts.

Examples:

A,B,C etc. = Special Comments

NOTE: = General information pertaining to that specific page

NLA = No Longer Available

HARDWARE MEASUREMENTS

The measurements for the hardware will be listed as follows:

Screws, Bolts Diameter X Thread Count/Pitch X Length

Washers Inside Diameter X Outside Diameter X Thickness

Spacers Inside Diameter X Outside Diameter X Length

For your convenience, we have supplied an area for you to list your Invacare Representatives.

Name	Phone Numbers

© 2008 Invacare Corporation, Elyria, Ohio. Invacare, the Medallion Design and Yes, you can. are registered trademarks of the Invacare Corporation.
Platinum is a trademark of Invacare Corporation. Information is subject to change without notification.

TABLE of CONTENTS

Description	Page Number
Cabinet and Base - Non Platinum 5 Models	4
Cabinet and Base - Platinum 5 Models	6
Compressor and Mounting Hardware - Non Platinum 5 Models	9
Compressor and Mounting Hardware - Platinum 5 Models	11
Control Panel and PC Board - Non Platinum 5 Models	13
Control Panel and PC Board - Platinum 5 Models	16
Packaging, Literature and Preventative Maintenance Kit	3
Sieve Bed Assembly and Product Tank - Non Platinum 5 Models	24
Sieve Bed Assembly and Product Tank - Platinum 5 Models	27
Sound Box Assembly - Non Platinum 5 Models	18
Sound Box Assembly - Platinum 5 Models	21

Packaging, Literature and Preventative Maintenance Kit




Note: When adjusting or replacing parts, please refer to your Owner's Manual for assistance.

Models	Packaging Kit	Carton	Top Insert	Poly Bag	Bottom Insert	Miscellaneous Pieces Assembly
Non-Platinum	1081290-NLA See Note(s) A	2000725	2001779	2000454	2001780	1081289 See Note(s) C
Platinum	1112296-NLA See Note(s) B	1108744	1108745	2000454	1108747	N/A
NOTE: A - Packaging Kit Includes: (1) each Carton, Top Insert, Bottom Insert, Poly Bag and See Note C. B - Packaging Kit Includes: (1) each Carton, Top Insert, Bottom Insert, Poly Bag and Owner's Manual. C - Miscellaneous Assembly Includes: (1) each Owner's Manual, Humidifier Adaptor, Humidifier Bottle Tubing and Poly Bag.						

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	1083917	Owner's Manual, International	1	1
2	A,C	1110827-NLA	Owner's Manual, International	1	1
4	A,B	2000895 See Chart	Kit, Preventative Maintenance (IRC3LX,5LX) Packaging and Literature	1 1	1 1
NOTE: A - Not Shown B - Non-Platinum 5 Models C - Platinum 5 Models					

Cabinet and Base

Non Platinum 5 Models

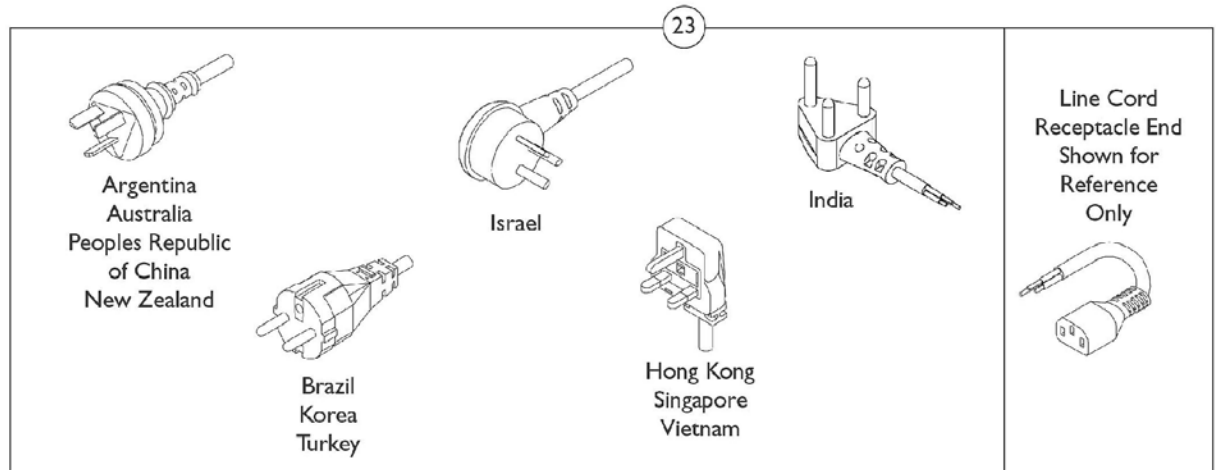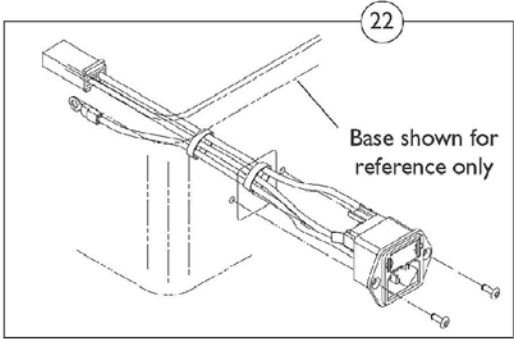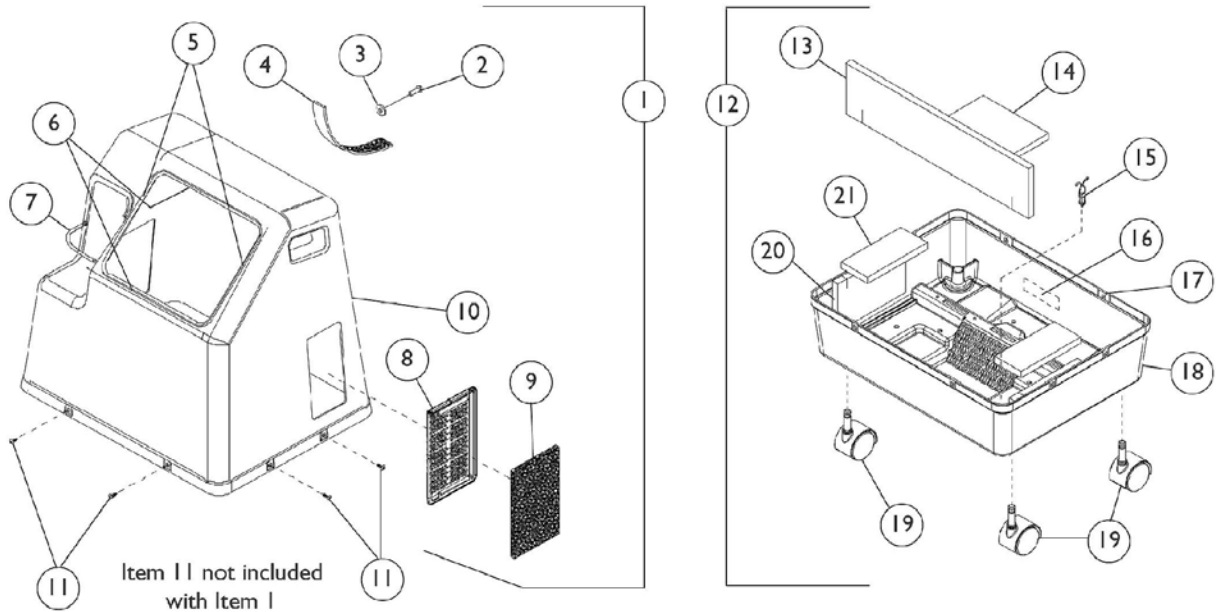



Cabinet and Base Non Platinum 5 Models

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	2002295	Cabinet Assembly	1	1
2		2002540	Rivet, Plastic	1	1
3		1022500	Retainer, Push On	1	1
4		P0875X071-NLA	Strap, Hook and Loop	1	1
5		2001838	Gasket, Foam (9-13/16")	1	2
6		2001839	Gasket, Foam (8")	1	2
7		1008547	Cord, Elastic	1	1
8		2002107	Grid, Filter	1	1
9		2000489	Filter, Cabinet	1	1
10		2002150	Top, Cabinet	1	1
11	B	2000508	Screw, Truss Head Self Tap (#8-32 x 5/8")	1	8
12		1083454	Assembly, Base	1	1
13		1089270	Foam, Base Front	1	1
14		1086642	Foam, Base (3/4")	1	1
15		2000576	Stand-Off, Twist-Lok (1/2")	1	1
16		1083465	Label, Specification	1	1
17		2000360	Grommet, Plastic	1	8
18		1075658	Base, Finished	1	1
19		1076830-NLA	Caster (2")	1	4
19		1110426	Caster (2")	1	4
20	C	2002000	Foam, Base Side	1	2
21		2001997	Foam, Base	1	2
22		2002574	Inlet Module Mounting	1	1
23		1062794	Cord, Line-Argentina, Australia, Peoples Republic of China and New Zealand	1	1
23		1062791-NLA	Cord, Line-Brazil, Korea and Turkey	1	1
23		1131308	Cord, Line, Continental Europe	1	1
23		1102634	Cord, Line-India	1	1
23		1071398	Cord, Line-Israel	1	1
23		1062793	Cord, Line-Hong Kong, Singapore and Vietnam	1	1
23		1071428	Label, Spanish (IRC5LX)	1	1
	D,E,F	1071430	Label, Spanish (IRC5LX02)	1	1
	D,E,F	1079322	Label, Brazilian Portuguese (IRC5LX)	1	1
	D,E,F	1079323	Label, Brazilian Portuguese (IRC5LX02)	1	1

NOTE: A - Includes: (1) each Cable Tie, Cabinet Front Foam, (2) each Cabinet Side Foam and Items 2-10.
B - Includes Items 13-21.
C - Includes (2) each Briv Rivet.
D - Not Shown.
E - Labels are not available for the following: Australia, Peoples Republic of China, Hong Kong, India, Israel, Korea, New Zealand, Singapore and Vietnam.
F - Standard equipment manufactured with full English label applied. Language specific Label Overlays used to customize equipment.
G - Item no longer available, refer to Part #1110426.
H - Item no longer available, refer to Part #1131308.

Cabinet and Base

Platinum 5 Models

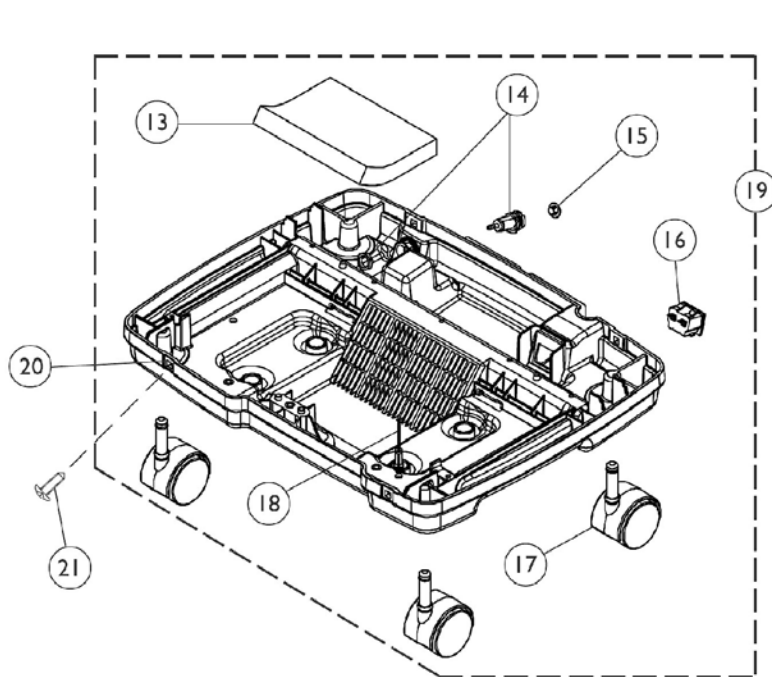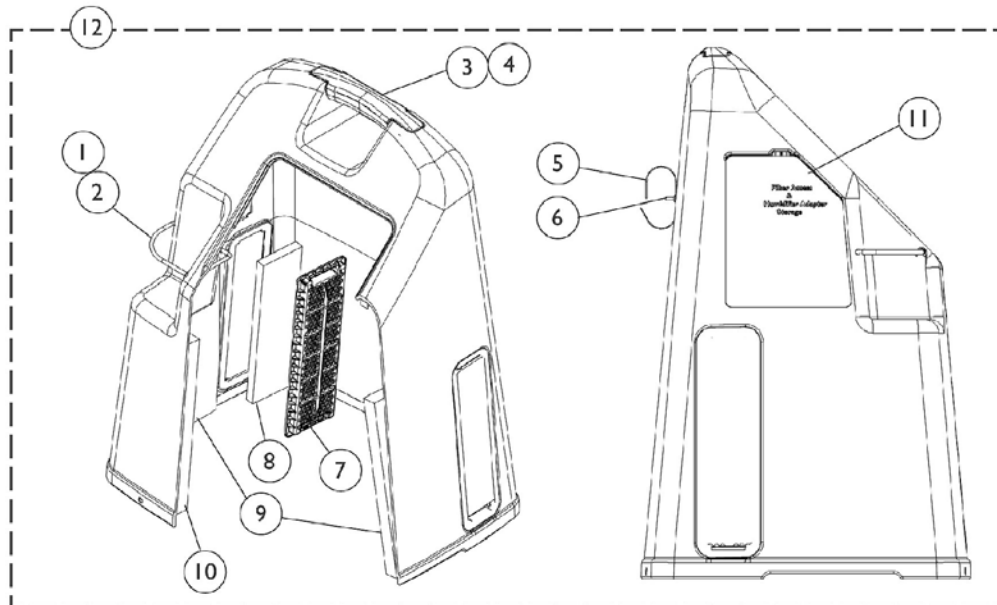

Cabinet and Base Platinum 5 Models

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1006525	Cable Tie (3-7/8" L)	1	1
2		1008547	Cord, Elastic	1	1
		1115868	Kit, Cabinet Handle	1	1
3			Handle, Top	1	1
4			Screw, Hex Head Tap (1/4-15 x 3/4")	1	2
	I	1153323	Kit, Strap for Power Cord w/ Hardware	--	--
			Strap, Hook and Loop	1	1
			Rivet, Plastic	1	1
			Retainer, Push On	1	1
5	H,I	P0875X071-NLA	Strap, Hook and Loop	1	1
6	H,I	2002540-NLA	Rivet, Plastic	1	1
7		1107400	Grid, Filter	1	1
8		1107412	Filter, Cabinet	1	1
9		1107414	Foam, Side Shroud	1	2
10		1107413	Foam, Front Shroud	1	1
11		1110766	Door, Filter Access with Foam Gasket	1	1
12	D,I	1108803	Cover, Cabinet Assembly (without Tie Wraps)	1	1
13		1108816	Foam, Base	1	1
14		1104151	Fitting, Hose Barb with Nut (1/8")	1	1
15		1108211	Cap, Dust	1	1
16		1105515	Inlet Module, 15A/ 250 VAC	1	1
17	G	1076830-NLA	Caster (2")	1	4
17		1110426	Caster (2")	1	4
18		2000576	Stand-Off, Twist-Lok (1/2")	1	1
19	E	1112259	Base, Cabinet Assembly	1	1
20		2000360	Grommet, Plastic	1	4
21		2000508	Screw, Truss Head Self Tap (#8-32 x 5/8")	1	4
22		1062794	Cord, Line-Argentina, Australia, Peoples Republic of China and New Zealand	1	1
22	F	1062791-NLA	Cord, Line-Brazil, Korea and Turkey	1	1
22		1131308	Cord, Line, Continental Europe	1	1
22		1102634	Cord, Line-India	1	1
22		1071398	Cord, Line-Israel	1	1
22		1062793	Cord, Line-Hong Kong, Singapore and Vietnam	1	1
	A,B,C	1071428	Label, Spanish (IRC5LX)	1	1
	A,B,C	1071430	Label, Spanish (IRC5LX02)	1	1
	A,B,C	1079322	Label, Brazilian Portuguese (IRC5LX)	1	1
	A,B,C	1079323	Label, Brazilian Portuguese (IRC5LX02)	1	1





Cabinet and Base

Platinum 5 Models

NOTE: A - Not Shown
B - Labels are not available for the following: Australia, Peoples Republic of China, Hong Kong, India, Israel, Korea, New Zealand, Singapore and Vietnam.
C - Standard equipment manufactured with full English label applied. Language specific Label Overlays used to customize equipment.
D - Includes Cover Cabinet and Items 1-11.
E - Includes Base Cabinet and Items 13-18.
F - Item no longer available, refer to Part #1131308.
G - Item no longer available, refer to Part #1110426.
H - Individual item no longer available, refer to Kit Part # 1153323 for replacement.
I - After 6/28/07, the Hook and Loop Strap are no longer used on the Cabinet Cover Assembly. If Hook and Loop Strap is needed you must order Kit, Strap for Power Cord w/ Hardware Part #1153323.

Compressor and Mounting Hardware

Non Platinum 5 Models

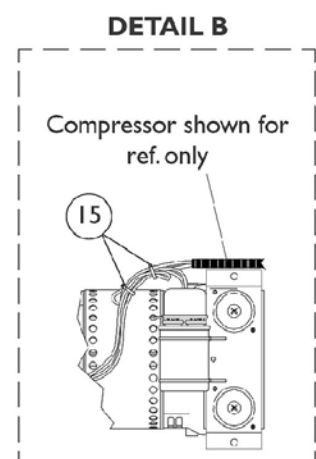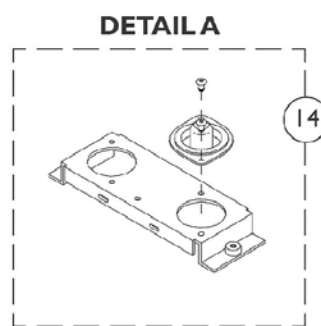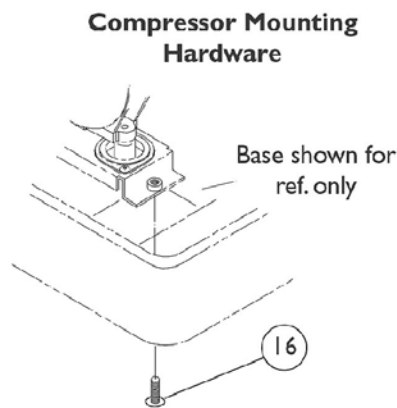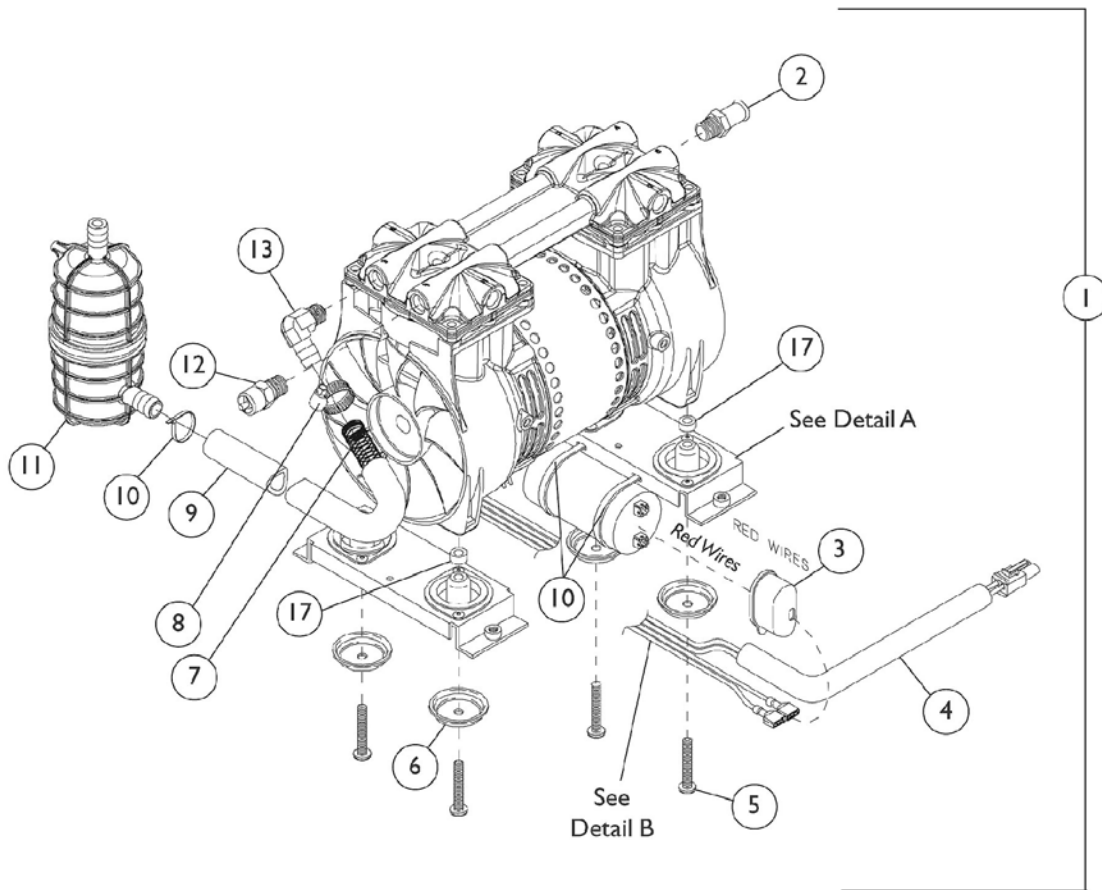

Compressor and Mounting Hardware



Non Platinum 5 Models

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	1101717	Assembly, Compressor	1	1
1	C,D,E	1108556	Assembly, Compressor	1	1
	F	1108481-NLA	Compressor, Ground Wire/Harness Kit	1	1
2		1086187	Hose Barbed, Patch Lock (1/2 x 1/4" NPT)	1	1
3		P0875X560	Boot, Terminal	1	1
4		1083224	Loom, Slit (17/64" I.D. x 12" Long)	1	1
5		1068064	Screw, Phillips Pan Head (1/4-20 x 1-3/4")	1	4
6		2000876	Washer, Cup	1	1
7		1088316	Spring, Extension (7/16 OD x 9-3/4" L)	1	1
8		P0876X026	Clamp, Hose	1	1
9		2002264	Tubing, Reinforced Braided (1/2 x 26")	1	1
10		2001682	Wrap, Tie (8")	1	3
11	G,K	1095537	Kit, Resonator Retro-Fit	1	1
			Resonator	1	1
			Screw, Phillips Pan Head (#10-16 x 1/2")	1	2
			Tubing, Silicon (1/2 x 5")	1	1
			Tube, Braided (10 x 1/2")	1	1
			Spring, Extension (7/16 OD x 9-3/4" L)	1	1
			Screw, Socket Button Head (1/4-20 x 3/8")	1	1
			Bracket, Mounting	1	1
			Clip, Flange	1	1
			Fitting, Elbow (1/2" Beaded Hose x 1/4" NPT)	1	1
			Wrap, Tie (8")	1	1
			Instruction Sheet	1	1
11	H	1085873	Resonator	1	1
12		1085697	Relief Valve with Patch Lock - 35 PSIG	1	1
13	G	2001374	Fitting, Resonator	1	1
13	H	1108819	Fitting, Elbow (1/2" Beaded Hose x 1/4" NPT)	1	1
14	I,M	2000888	Bracket Assembly, Mounting	1	2
15	N	1006525	Cable Tie (3-7/8" L)	1	2
16		2001795	Screw, Phillips Pan Head (1/4 X 1/2")	1	4
17	I,J,N	2002906	Spacer (1/4 x 1/2 x 7/32")	1	4

NOTE: A - Before 6/27/00.
 B - Includes Items 2-13 and Plastic Resonator Mounting Hardware.
 C - After 6/26/00.
 D - Includes Items 2-14.
 E - Units built before 4/10/00, Resonator Retro Fit Kit (#1095537) is required to make Compressor Assembly (#1080183) compatible with concentrator.
 F - Not Shown.
 G - Before 4/10/00.
 H - After 4/9/00.
 I - When ordering Bracket Assembly (#2000888) for 5LX02AW units built prior to 4/27/00, order Spacer (#2002906). After 4/26/00, Spacer was added as a product improvement on 5LX02AW models.
 J - After 4/26/00.
 K - Includes Resonator and Mounting Hardware.
 M - Includes Mounts and Hardware.
 N - Use with IRC5LX02.

Compressor and Mounting Hardware

Platinum 5 Models

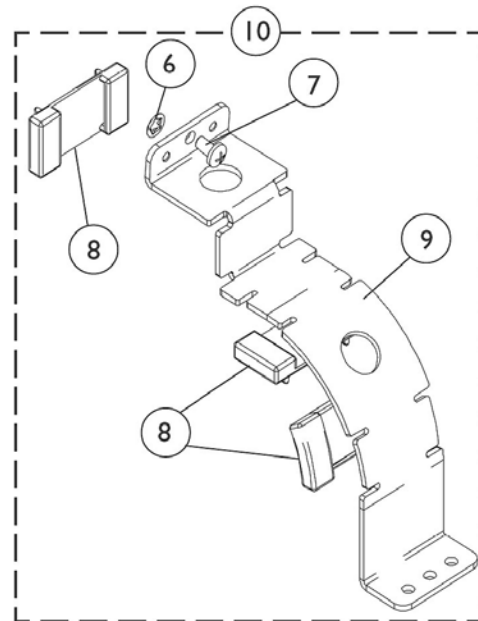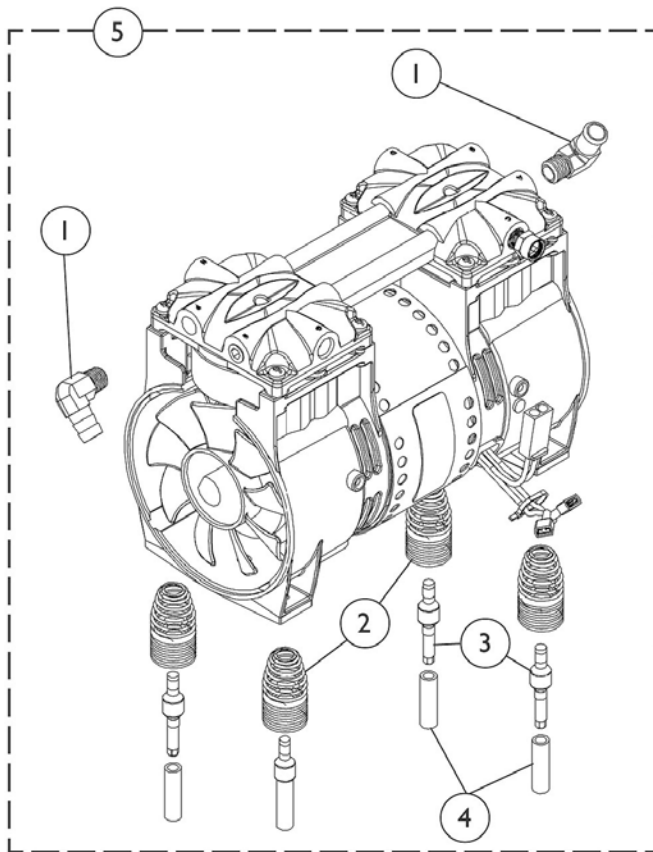


Compressor and Mounting Hardware Platinum 5 Models

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1108819	Fitting, Elbow (1/2" Beaded Hose x 1/4" NPT)	1	2
2		1105504-NLA	Spring, Compressor Mount	1	4
2		1150581	Spring, Compressor Mount	1	4
3		1106446-NLA	Pin, Spring Retainer	1	4
3		1150580	Pin, Spring Retainer	1	4
4		1107411	Glide, Pin	1	4
5		1110825	Compressor with Mounting Hardware	1	1
6		1108740	Retainer, Push (1/4")	1	1
7		1085696-NLA	Screw, Truss Head (1/4-20 x 1/2")	1	1
7		1128406-NLA	Screw, Phillips Hex Washer Head w/ Patch (1/4-20 x 3/4")	1	1
8		1108824	Bumper, Bracket	1	3
9		1107406	Bracket, Restraint (Thomas)	1	1
10		1108092	Bracket Restraint Assembly (Thomas)	1	1

Control Panel and PC Board

Non Platinum 5 Models

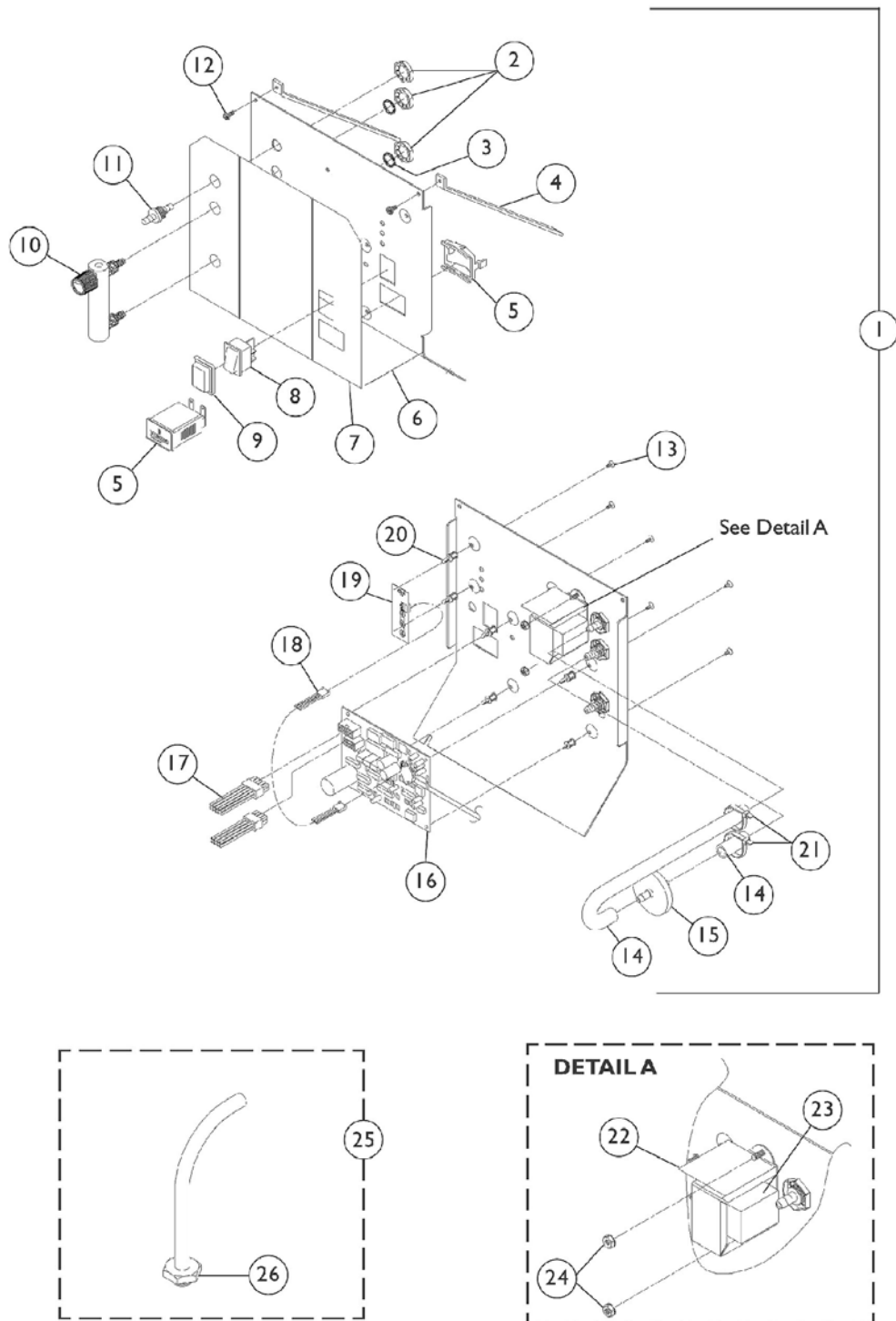

Control Panel and PC Board

Non Platinum 5 Models

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1102874-NLA	Assembly, Control Panel (IRC5LX)	1	1
1	B	1102875-NLA	Assembly, Control Panel Assembly (IRC5LX02)	1	1
2		2000507	Palnut (9/16-18)	1	3
3		2001139	O-Ring (1/2" I.D.)	1	2
4		2000464	Support, Control Panel	1	2
5		2002939	Hour Meter w/Clip	1	1
6		2002811-NLA	Control Panel	1	1
7	C	1081282	Label, English (IRC5LX)	1	1
7	C	1081283	Label, English (IRC5LX02AW)	1	1
7	C	1071428	Label, Spanish (IRC5LX)	1	1
7	C	1071430	Label, Spanish (IRC5LX02)	1	1
7	C	1079322	Label, Brazilian Portuguese (IRC5LX)	1	1
7	C	1079323	Label, Brazilian Portuguese (IRC5LX02)	1	1
7	D	1102877	Label, Control Panel, Logo	1	1
7	D	1102876	Label, Control Panel, English (IRC5LX)	1	1
7	D	1102878	Label, Control Panel, English (IRC5LX02)	1	1
7	D	1102921	Label, Control Panel, English (IRC5LX)	1	1
7	D	1102922	Label, Control Panel, Spanish (IRC5LX02)	1	1
7	D	1102923	Label, Control Panel, Brazilian Portuguese (IRC5LX)	1	1
7	D	1102924	Label, Control Panel, Brazilian Portuguese (IRC5LX02)	1	1
8		2002875-NLA	Switch, On/Off	1	1
9		2001141	Seal, Switch	1	1
10	E	2002839-NLA	Flow Meter, 0-5 LPM	1	1
	G	1071275	Kit, Set-Up Pediatric Flowmeter	1	1
			Tubing, Oxygen Supply (21 Feet)	1	1
			Tubing, Oxygen Supply (7 Feet)	1	2
			Connector, Tubing	1	1
			Adapter, Humidifier Bottle	1	1
			Instruction Manual	1	1
11		2000473	Connector, Barbed (1/4")	1	1
12		2003065	Screw, Phillips Hex Washer Head (#8-32 x 3/8")	1	2
13		1028794	Screw, Self Tapping (#6 x 1/4")	1	6
14		2000897	Tubing (1/4 x 24")	1	1
15		P0750X030	Filter, HEPA Outlet	1	1
16		1069477	P.C. Board (IRC5LX)	1	1
16		1079446	P.C. Board (IRC5LX02)	1	1
17		2002577	Harness, Wiring (24V)	1	1
18		2001762	Harness, Wire-Sensor (IRC5LX02)	1	1
19		1061773	Remote LED Assembly - Sens 02 (IRC5LX02)	1	1
20		1036162	Stand-Off, P.C. Board	1	6
21		2001682	Wrap, Tie (8")	1	2
22		2002553	Insulator, Transformer	1	1



Control Panel and PC Board

Non Platinum 5 Models

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
23	F	1105391	Transformer Kit	1	1
24		P0870X050	Nut, Keps (#6-32)	1	2
25		2000799	Adapter with Tubing for Humidifier Bottle	1	1
26		2000503	Adapter, Humidifier Bottle	1	1
NOTE: A - Includes items 2-17 and 20-24. B - Includes items 2-24. C - Before 6/28/01. D - After 6/27/01. E - Includes Item 2. F - Includes Insulator and Kep Nuts. G - Not Shown.					

Control Panel and PC Board

Platinum 5 Models

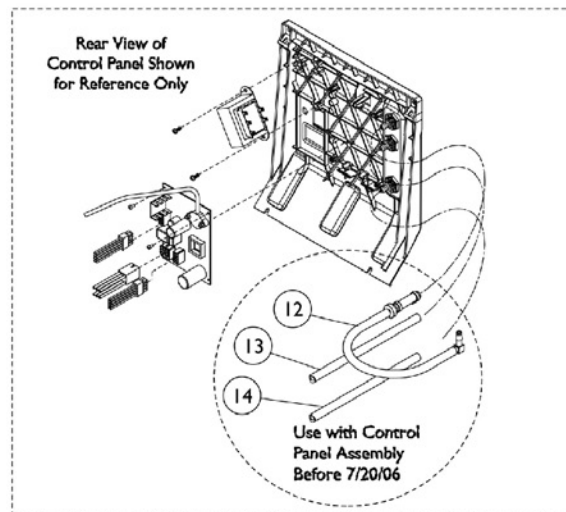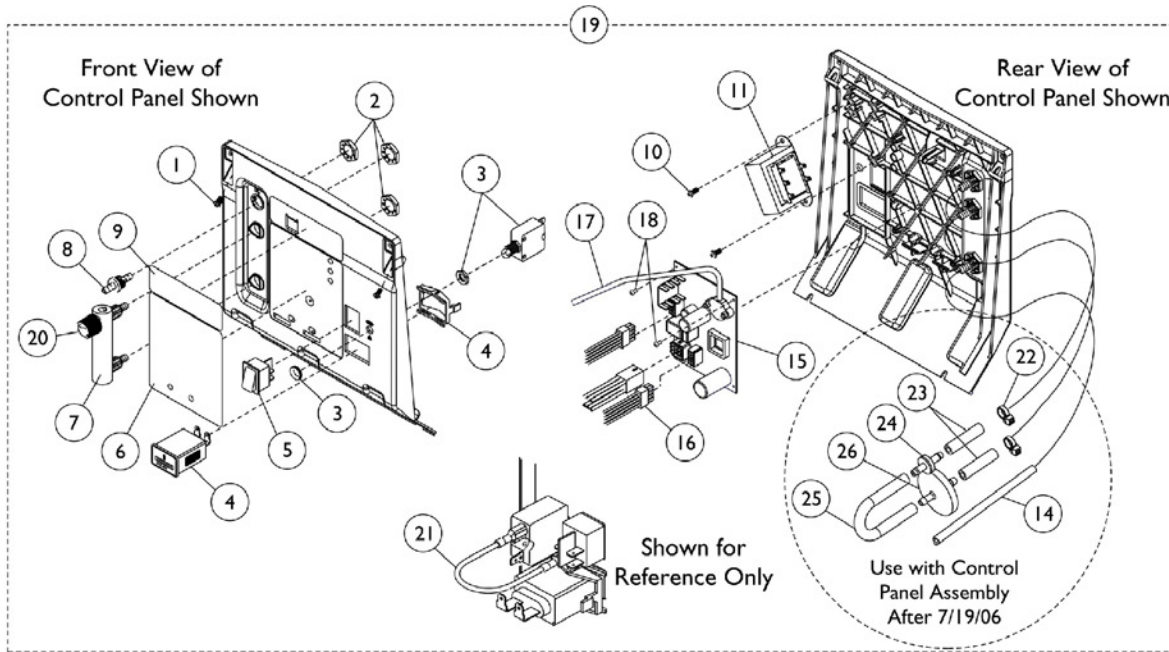


1128379



Used with Flowmeters that have an Indented Phillips Head Screw on top.

1128370

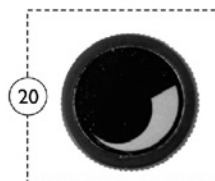

Used with Flowmeters that have a Slotted Head Screw on top.


Control Panel and PC Board Platinum 5 Models

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		2003065-NLA	Screw, Phillips Hex Washer Head (#8-32 x 3/8")	1	2
1		1128404	Screw, Phillips Hex Head (#8-32 x 3/8")	1	2
2		2000507	Palnut (9/16-18)	1	3
3		1110824	Circuit Breaker, 4 Amp.	1	1
4		2002939	Hour Meter w/Clip	1	1
5		2001134	Switch, On/Off	1	1
6		1110835	Decal, Control Panel, IRC5LX	1	1
6		1110830	Decal, Control Panel, IRC5LX02	1	1
7		1110823-NLA	Flow Meter, 0-5 LPM	1	1
7		1110561	Flowmeter without Palnuts, 5 Liter	1	1
8		2000473	Connector, Barbed (1/4")	1	1
9		1108757	Decal, Logo	1	1
10		1102696	Screw, Phillips Pan Head (#8-32 x 3/8")	1	2
11	H	1105385-NLA	Transformer	1	1
	I	1105391	Transformer Kit	1	1
12	A,G	1110779	Tubing Assembly, Check Valve Discharge	1	1
13	G	1108815	Tubing, Silicon (1/4 x 16-1/4")	1	1
14		1108809	Tubing, PVC (1/4 x 12-3/4")	1	1
15	B	1069477	P.C. Board (IRC5LX)	1	1
15		1079446	P.C. Board (IRC5LX02)	1	1
16		1110815	Wire Harness, 10 Pin	1	1
17		1076275	Tubing (1/8 x 18")	1	1
18		2001902	Screw, Phillips Pan Head (#4-40 x 1/4")	1	2
19		1110836	Assembly, Control Panel (IRC5LX)	1	1
19		1110831-NLA	Assembly, Control Panel (IRC5LX02)	1	1
20	D	1128379	Knob, Flowmeter Silver with Black Swish	10	1
20	C	1128380	Knob, Flowmeter Black with Silver Swish	10	1
21		2001764	Jumper Wire Assembly	1	1
22	E,F	2001682	Wrap, Tie (8")	1	2
23	E,F	1110473	Tubing, PVC (1/4 x 1-3/4")	1	2
24	E,F	2000460	Valve, Check	1	1
25	E,F	1143144	Tubing, PVC (1/4 x 5")	1	1
26	E,F	P0750X030	Filter, HEPA Outlet	1	1


NOTE: A - Includes (2 each) Tubing, Check Valve, Elbow and Tie Wrap.
B - Includes Item 17.
C - Knob is black with silver "swish". Used on Flowmeters with a Slotted Head Screw on top.
D - Knob is silver with black "swish". Used on Flowmeters with an Indented Phillips Head Screw on top.
E - "After 7/19/06" the location of the HEPA Filter has changed. The HEPA Filter is now located inside the cabinet.
F - For use with Control Panel Assembly "After 7/19/06."
G - For use with Control Panel Assembly "Before 7/20/06."
H - Individual item no longer available, refer to Transformer Kit Part #1105391 for replacement.
I - Includes Transformer, Nuts and Insulator.

Sound Box Assembly


Non Platinum 5 Models

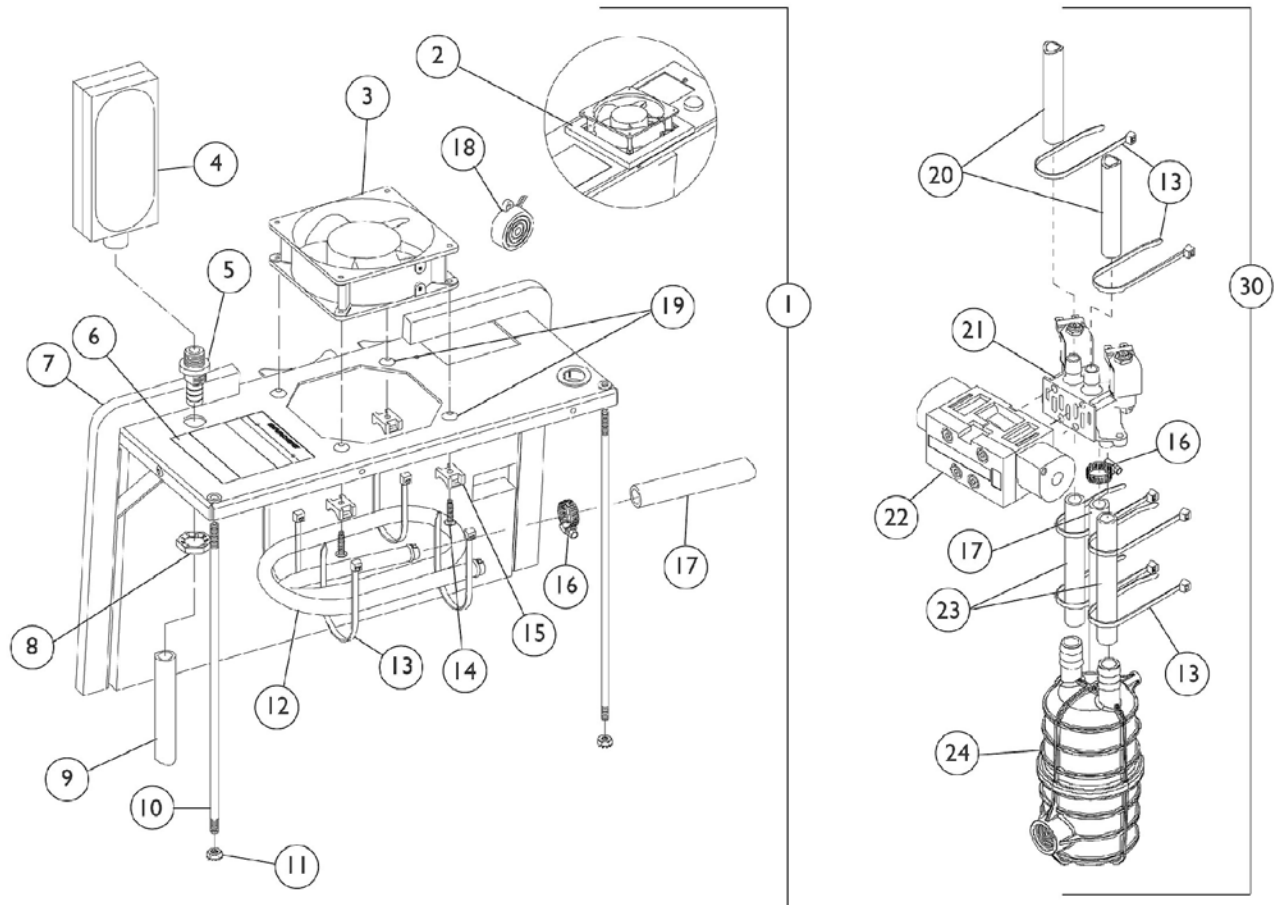

**Valve, Manifold,
and Muffler**

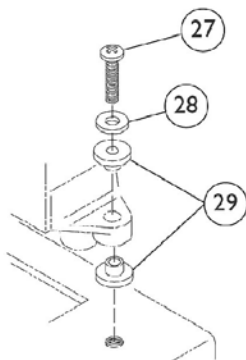

Manifold Mounting Hardware

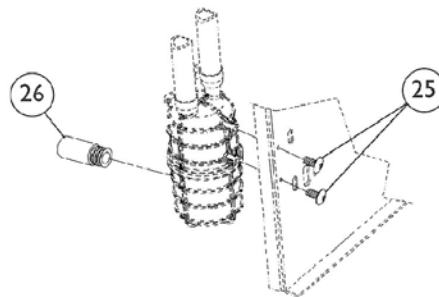

Muffler Mounting Hardware

Sound Box Assembly

Non Platinum 5 Models


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	B	1080182-NLA	Sound Box Top Assembly	1	1
2		1073613	Gasket, Foam	1	1
3		1071657	Fan, Exhaust	1	1
4	K	1108078-NLA	Filter, Inlet	1	1
4		1131249	Filter, Inlet Sound Reduced	1	1
5		2001096	Fitting, Bulk Head (1/2")	1	1
6		2002997	Label, Diagnostic (IRC5LX02)	1	1
7		2001802-NLA	Sound Box Top w/Brackets	1	1
8		P0875X720	Nut (3/4-16)	1	1
9	C	2000903	Tubing, Braided (1/2 X 24")	1	1
9	D	1088281	Tubing, Silicon (1/2 x 5")	1	1
	A	2001194-NLA	Spring, Compression (17-1/4")	1	1
10	C	2000411	Support Rod	1	2
11		P0870X030	Nut, Keps (1/4-20)	1	2
12		1085014	Heat Exchanger	1	1
13		2001682	Wrap, Tie (8")	1	10
14		1076750	Screw, Sheet Metal (10-24 x 3/4")	1	4
15		2000755	Mount, Cable Tie	1	4
16		P0876X026	Clamp, Hose	1	1
17		2001375	Hose, Reinforced Silicon (1/2 x 8-1/2")	1	1
	A	2000765	Spring, Extension (7/16 OD x 7-1/2" L)	1	1
18		2001765	Harness, Horn/Alarm	1	1
19	E	2000834	Grommets	1	4
20		2002264	Tubing, Reinforced Braided (1/2 x 26")	1	2
21		1105552	Kit, 4 Way Valve/Manifold Service	1	1
			Manifold Assembly	1	1
			Valve, 4-Way with Screws	1	1
			Hose, Reinforced Silicon (1/2 x 8-1/2")	1	1
			Tubing, Reinforced Braided (1/2x 4-1/8")	1	2
			Clamp, Hose	1	1
			Tubing, Reinforced Braided (1/2 x 26")	1	2
			Wrap, Tie (8")	1	6
			Clamp, Plastic (1/2")	1	2
			Bushing, Step (1/2")	1	4
			Washer, Flat (#8)	1	2
			Screw, Phillips Pan Head (#8-32 x 3/4")	1	2
			Resistor Service Assembly	1	1
			Tie, Cable	1	2
			Instruction Sheet	1	1
22		1083005-NLA	Valve, 4-Way with Screen	1	1
23	C	2002265	Tubing, Reinforced Braided (1/2 x 5-1/4")	1	2
23	D	1095209	Tubing, Reinforced Braided (1/2x 4-1/8")	1	2
24	D,H	1095538	Kit, Muffler Retro-Fit	1	1

Sound Box Assembly Non Platinum 5 Models

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
			Muffler Assembly	1	1
			Screw, Phillips Pan Head (#10-16 x 1/2")	1	2
			Screw, Socket Button Head (1/4-20 x 3/8")	1	1
			Bracket, Mounting	1	1
			Clip, Flange	1	1
			Instuction Sheet	1	1
25	C	1085696	Screw, Truss Head (1/4-20 x 1/2")	1	1
25	D	1087225	Screw, Phillips Pan Head (#10-16 x 1/2")	1	2
26		2001116	Uni-Muffler	1	1
27		1090138	Screw, Phillips Pan Head (#8-32 x 3/4")	1	2
28		P0870X490	Washer, Flat (#8)	1	2
29		2001815	Bushing, Step (1/2")	1	4
30	F,G	2001803-NLA	Valve Sub-Assembly	1	1
	A,I	1080971	Kit, Pilot Coil Replacement	1	1
			Coil, Pilot	1	1
			Instruction Sheet, Pilot Coil	1	1
	A,J	1080973	Kit, Pilot Valve Stem and Poppet Replacement	1	1
			Pilot Valve Poppet and Stem	1	1
			Instruction Sheet, Pilot Coil	1	1
NOTE: A - Not Shown. B - Includes Items 2-19. C - Before 4/10/00. D - After 4/9/00. E - After 7/31/00. F - Includes 13, 16-17 and 20-24. G - When ordering Valve Sub-Assembly for units built before 4/10/00, it is required the Muffler Retro Kit (#1095538) be ordered as well. H - Includes Muffler and Mounting. I - Includes Pilot Coil and Instruction Sheet. J - Includes Pilot Valve Poppet and Instruction Sheet. K - Item no longer available, refer to Part #1131249.					

Sound Box Assembly

Platinum 5 Models

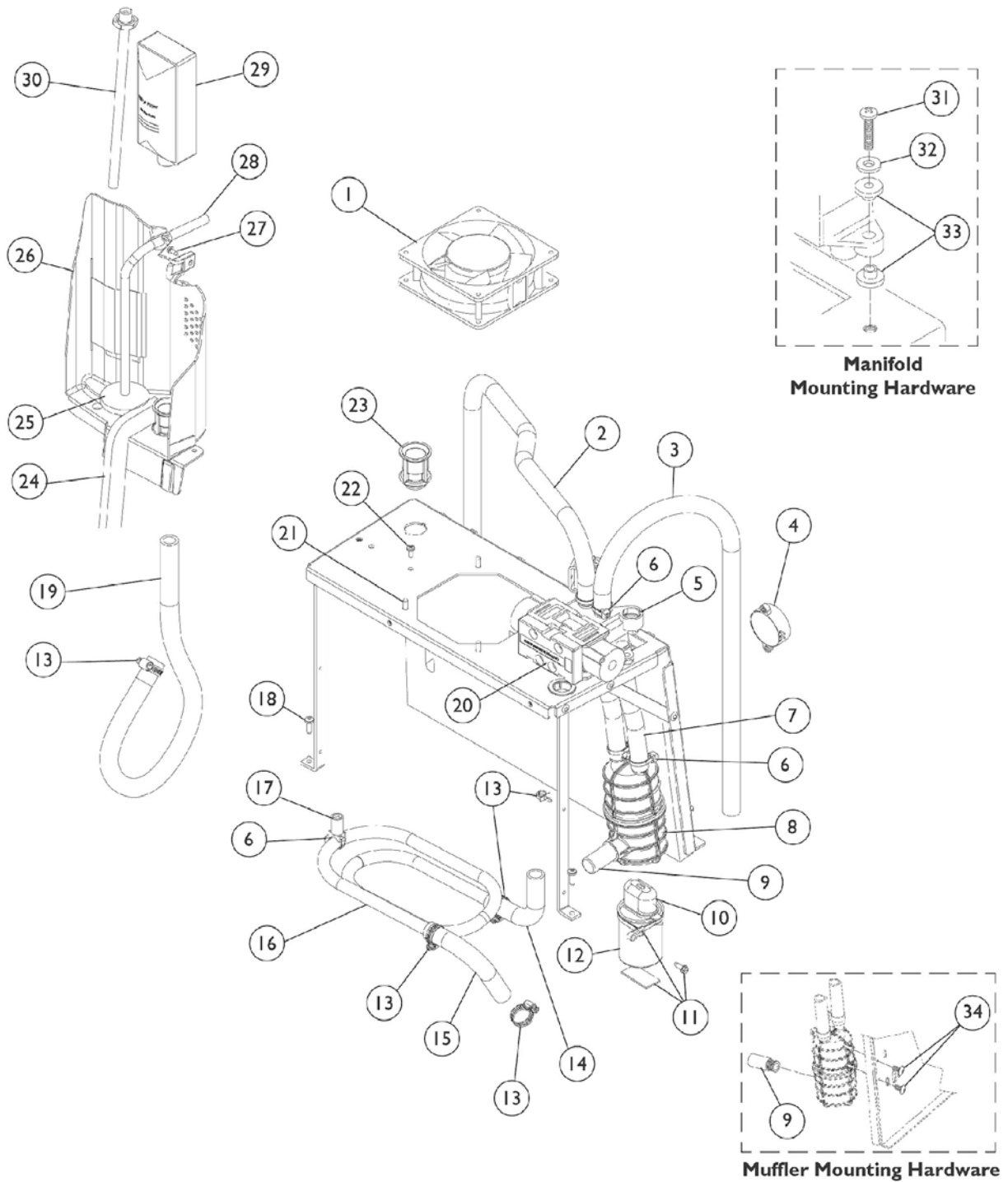

Sound Box Assembly

Platinum 5 Models







Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1071657	Fan, Exhaust	1	1
2		1108783	Tubing (1/2 x 23")	1	1
3		1108784	Tubing (1/2 x 20-1/4")	1	1
4		2001765	Harness, Horn/Alarm	1	1
5	A,B	1108085	Manifold/Valve Assembly	1	1
6		2001682	Wrap, Tie (8")	1	5
7		1110848	Tubing (1/2 x 6-3/16")	1	2
8		1085872	Muffler Assembly	1	1
9		2001116	Uni-Muffler	1	1
10		P0875X560	Boot, Terminal	1	1
11	C	1110776-NLA	Capacitor Mounting Hardware	1	1
12		1068698	Capacitor, 15 MFD	1	1
13		P0876X026	Clamp, Hose	1	5
14		1108822	Elbow, PVC Formed (1/2 x 2-1/2 x 2-1/2")	1	1
15		1106698	Elbow, Silicon (1/2 x 3-1/2")	1	1
16	G	1107422-NLA	Heat Exchanger	1	1
	E,H	1108817	Heat Exchanger Assembly	1	1
17		1110780	Stand-Off, Heat Exchanger	1	1
18		1108810	Screw, Hi-Lo Six Lobe Drive (#10-16 x 5/8")	1	7
19		1108086-NLA	Tubing (1/2 x 19")	1	1
19		1112561	Tube, Compressor	1	1
20		1101141	Valve, 4-Way with Screws	1	1
	E	1101141R	Valve, Reconditioned	1	1
	E	1101141E	Kit, Valve - Warranty Replacement	1	1
			Valve, Reconditioned	1	1
			Gasket	1	1
			Screw, Socket Head Cap (M5 x 51/64)	1	4
21		1110852	Screw, Phillips Pan Head (#10-24 x 1/2")	1	4
22		1003363-NLA	Screw, Phillips Pan Head (#8-32 x 3/8")	1	1
22		1128404	Screw, Phillips Hex Head (#8-32 x 3/8")	1	1
23		1106675	Coupling, Inlet Filter	1	1
24		2000812	Seal, Foam	1	2
25	F	P0750X030	Filter, HEPA Outlet	1	1
26		1110777-NLA	Partition	1	1
26		1115183	Partition	1	1
27		2003065-NLA	Screw, Phillips Hex Washer Head (#8-32 x 3/8")	1	1
27		1128404	Screw, Phillips Hex Head (#8-32 x 3/8")	1	1
28		1108815	Tubing, Silicon (1/4 x 16-1/4")	1	1
29	D	1108078-NLA	Filter, Inlet	1	1
29		1131249	Filter, Inlet Sound Reduced	1	1
30		1110768-NLA	Adaptor, Humidifier Bottle	1	1
30		2000799	Adapter with Tubing for Humidifier Bottle	1	1
31		1090138	Screw, Phillips Pan Head (#8-32 x 3/4")	1	2

Sound Box Assembly Platinum 5 Models

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
32		P0870X490	Washer, Flat (#8)	1	2
33		2001815	Bushing, Step (1/2")	1	4
34		1087225	Screw, Phillips Pan Head (#10-16 x 1/2")	1	2
NOTE: A - Manifold is not available as a separate item. Must order item 5, Manifold/Valve Assembly. B - Includes Items 2-3, 6-8, 13, 14 and 20. C - Includes Screw, Tie Wraps and Foam Tape. D - Item no longer available, refer to Part #1131249. E - Not Shown. F - After 7/19/06 the Hepa Filter was moved to the inside of cabinet, located between the Flow Meter and the Patient Outlet Barb. See "Control Panel and P.C. - Plat. 5 " category to view reflective changes. G - Item no longer available, refer to part #1108817 for replacement. H - Includes Heat Exchanger, Standoff, and Tie Wrap.					

Sieve Bed Assembly and Product Tank

Non Platinum 5 Models

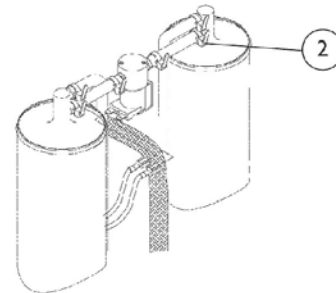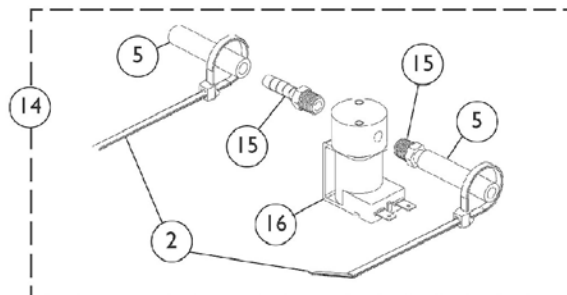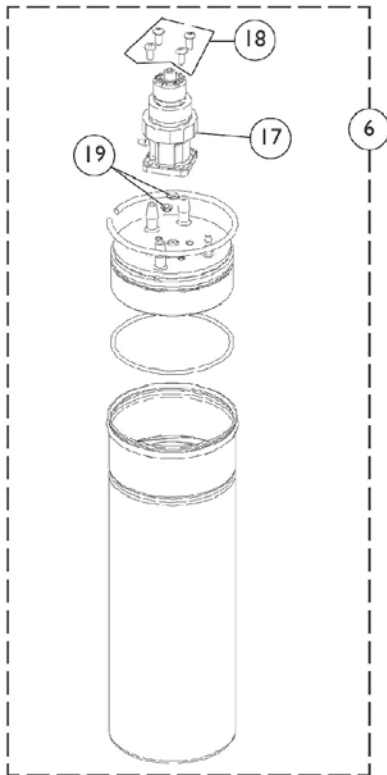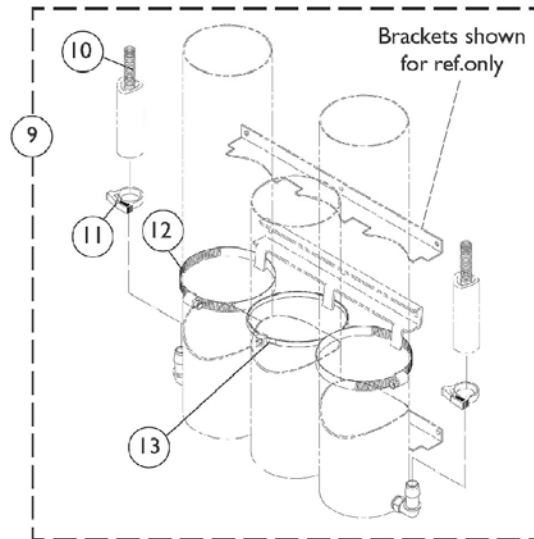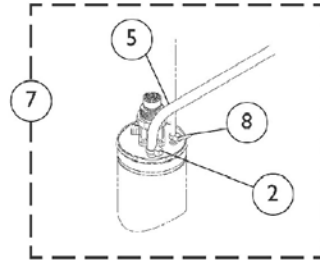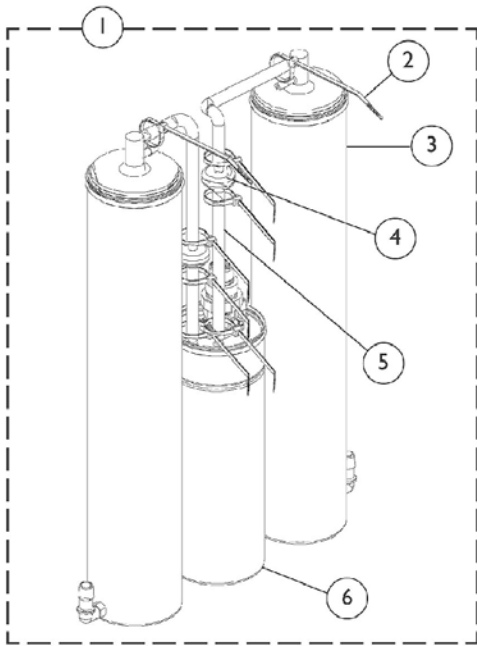

Sieve Bed Assembly and Product Tank

Non Platinum 5 Models

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1102839-NLA	Sieve Bed/Product Tank Assembly (IRC5LX)	1	1
1	A,N	1102838-NLA	Sieve Bed/Product Tank Assembly (IRC5LX02)	1	1
1	O	1110783	Sieve Bed Assembly (IRC5LX02)	1	1
2		2001682	Wrap, Tie (8")	1	8
3	B	1101707	Sieve Bed Assembly	2	1
4		2000436	Check Valve, Barb (5/16")	1	2
5		2000897	Tubing (1/4 x 24")	1	1
6	C	1082588	Product Tank (IRC5LX)	1	1
6	C	1075665	Product Tank (IRC5LX02)	1	1
7	D	1082587	Product Tank Mounting Hardware (IRC5LX)	1	1
7	D	1076216	Product Tank Mounting Hardware (IRC5LX02)	1	1
8		1070050	Wrap, Tie	1	2
9	E	2001806-NLA	Sieve Bed/Product Tank Mounting Hardware	1	1
10		2002391	Spring, Compression (13/32 x 28")	1	2
11		2001652	Clamp, Plastic (1/2")	1	2
12		1002807	Clamp, Hose	1	2
13		1089455	Wrap, Tie (15-1/4")	1	4
14	F	2001656	PE Valve Assembly w/ Fitting	1	1
	G,K	1080970	Kit, PE Coil Replacement	1	1
			PE Coil	1	1
			Kit, PE Coil/Valve Instruction Sheet	1	1
	K,L	1080972	Kit, PE Valve Stem and Poppet Replacement	1	1
			PE Valve Poppet and Stem	1	1
			Kit, PE Coil/Valve Instruction Sheet	1	1
	M	1124855	Kit, PE Valve with Fittings	1	1
15			Fitting, Barb (1/4 x 1/8")	2	1
16			PE Valve, Coil (24V)	1	1
17		1081381	Regulator	1	1
17		1075506	Regulator	1	1
18		1075666	Screw, Regulator (8-32 x 3/8")	1	4
19		2002590	O-Ring (11/64" I.D.)	1	2
	I,K	2001671	Kit, Pressure Gauge Tap In without Gauge	1	1
			Fitting, Tee Barb (1/4")	1	1
			Plug, Hose (1/4")	1	1
			Clamp (1/4")	1	4
			Tubing, Silicone (1/4 x 16")	1	1
			Instruction Sheet	1	1
	J,K,P	2001673-NLA	Kit, Pressure Gauge Tap In with Gauge	1	1
			Fitting, Tee Barb (1/4")	1	1
			Plug, Hose (1/4")	1	1
			Clamp (1/4")	1	4
			Tubing, Silicone (1/4 x 16")	1	1


Sieve Bed Assembly and Product Tank

Non Platinum 5 Models

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
		2001674	Kit, Pressure Gauge	1	1
			Instruction Sheet	1	1
			Kit, Pressure Gauge	1	1
NOTE: A - Includes items 2-6. B - Includes (2) Sieve Beds, (2) Clamps, (2) Hose Clamps and (4) Tie Wraps. C - Includes items 17-19. D - Includes items 2, 5 and 8. E - Includes items 10-13. F - Includes items 2, 5, 15 and 16. G - Includes PE Coil and Instruction Sheet. I - Includes Clamps, Tubing, Hose Plug, Fitting and Instruction Sheet. J - Includes Clamps, Tubing, Hose Plug, Fitting, Gauge and Instruction Sheet. K - Not Shown. L - Includes PE Valve Poppet and Instruction Sheet. M - Includes items 15-16. N - Item no longer available, refer to Part #1110783. O - Includes: (2) Sieve Beds, (4) Hose Clamps and (4) Tie Wraps. P - Item no longer available. Please order part #2001671 and 2001674 as a replacement.					

Sieve Bed Assembly and Product Tank

Platinum 5 Models

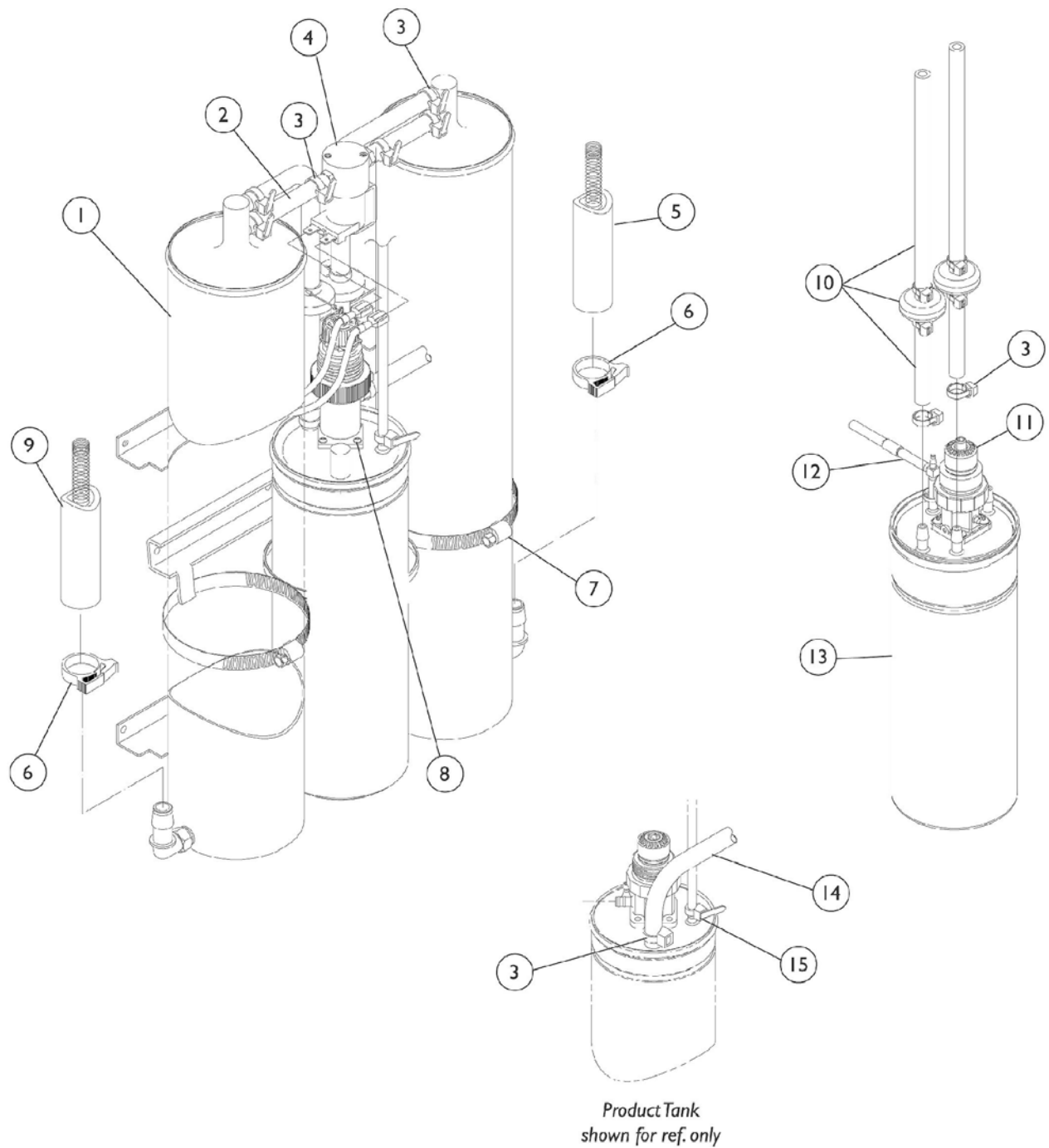

Sieve Bed Assembly and Product Tank

Platinum 5 Models

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1101707	Sieve Bed Assembly	2	2
2		1110774	Tubing, PVC (1/4 x 2-1/2")	1	2
3		2001682	Wrap, Tie (8")	1	9
4	B	2001656	PE Valve Assembly w/ Fitting	1	1
5		1108783	Tubing (1/2 x 23")	1	1
6		2001652	Clamp, Plastic (1/2")	1	2
7		1002807	Clamp, Hose	1	2
8		1070147	Screw, Regulator (#4-24 x 1")	1	4
9		1108784	Tubing (1/2 x 20-1/4")	1	1
10	C,F	1110474-NLA	Check Valve Assembly	1	2
10		1104310	Kit, Check Valve	1	2
			Clamp, Hose	1	2
			Wrap, Tie	1	1
			Instruction Sheet	1	1
			Check Valve, Barb (5/16")	1	2
			Wrap, Tie (8")	1	9
			Tube (1/4")	1	1
11		1081381	Regulator	1	1
11		1075506	Regulator	1	1
12	D	1106774	HomeFill II Tubing Assembly	1	1
13	E	1082588	Product Tank (IRC5LX)	1	1
13	E	1075665	Product Tank (IRC5LX02)	1	1
14		1108809	Tubing, PVC (1/4 x 12-3/4")	1	1
15		1070050	Wrap, Tie	1	1
NOTE: A - Includes (2) Sieve Beds, (2) Clamps, (2) Hose Clamps and (4) Tie Wraps. B - Includes items 2-3, PE Valve and Barb Fitting. C - Includes Check Valve, Tubing and Tie Wraps. D - Includes Tubing and Fittings. E - Includes items 8 and 11. F - Individual item no longer available, refer to Kit Part#1104310 for replacement.					

Invacare Corporation

www.invacare.com



Yes, you can.™

One Invacare Way
Elyria, Oh USA
44036-2125
800-333-6900

Technical Support
800-832-4707
Florida Technical Support
407-321-5630

Customer Service
800-333-6900

© 2008 Invacare Corporation, Elyria, Ohio.
All rights reserved. Trademarks are
identified by the symbols ™ and ®. All
trademarks are owned by or licensed to
Invacare Corporation unless otherwise
noted. Palnut is a registered trademark of
Transtechonology Corporation. Information is
subject to change without notification.
(Revised 05/22/08)

Form No. 99-176