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Italiano

1. Legend

1. Handle for lifting or moving
2. On/Off knob
3. Warning notices
4. Operating hours meter
5. Oxygen outlet
6. Connecting nipple with union nut (fitted)
7. Connecting tube (20 m)
8. Castors with brake
9. Dosage monitor
10. Power cord
11. Connecting tube (3 m) for nasal cannula
12. Nasal cannula
13. Coarse dust filter
14. Power cord socket
15. Fuse holder
16. Flowmeter
17. Connecting tube (10 m)
18. Fault indicator (red)
19. Oxygen concentration status indicator (yellow)
20. Power indicator (green)
21. Dosage unit (flowmeter)
22. Ball, indicates flow
23. Connection, long tube to OXYMAT 3 (20 m)
24. Flow control knob
25. Humidifier glass of bubble humidifier
26. Connection, 3 m hose to nasal cannula
27. Flow control knob
28. Connection, OXYMAT 3
29. Connection for tubes (3 m or 10 m)

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Español

Contents

1. Legend	68
2. Description	74
2.1 Purpose	74
2.2 Function	76
3. Safety Information	78
4. Installation	84
4.1 Setting up device	84
4.2 Connecting to dosage monitor and nasal cannula	86
4.3 Connecting to flowmeter and nasal cannula	88
5. Operation	90
5.1 Before switching on	90
5.2 Switching on and off	90
5.3 Indicators	92
5.4 Storing OXYMAT 3	94
5.5 Bringing the OXYMAT 3 back into service	96
5.6 Tips	96
6. Hygienic preparation	98
6.1 Time periods	98
6.2 Preparation	100
6.3 Cleaning and disinfection	102
6.4 Sterilisation of sterile water tank	104
6.5 Assembly	104
6.6 Change in patients	104
7. Functional Check	106
8. Troubleshooting	110
8.1 General	110
8.2 Changing fuses	114
9. Maintenance	116
10. Disposal	118
11. Product and Spare Parts	120
11.1 Standard products	120
11.2 Spare parts	126

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2. Description

2.1 Purpose

Your doctor can tell you whether you need to keep a reserve oxygen supply for emergencies.

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The OXYMAT 3 oxygen concentrator is designed to increase the oxygen content (O_2) in the air that you breathe.

Oxygen therapy is an important medical measure. For many serious disorders of the respiratory organs and cardiovascular system it is the only meaningful therapy possible. Oxygen (O_2) therapy is the most commonly used prophylactic or therapeutic measure in cases of impending or already existing hypoxia.

If you need continuous administration of air with an increased oxygen content, a reserve supply from an oxygen cylinder must always be assured in case of emergency (e.g. power failure) (see "5.6 Tips" page 96). Ask your doctor whether an emergency supply is necessary in your particular case.

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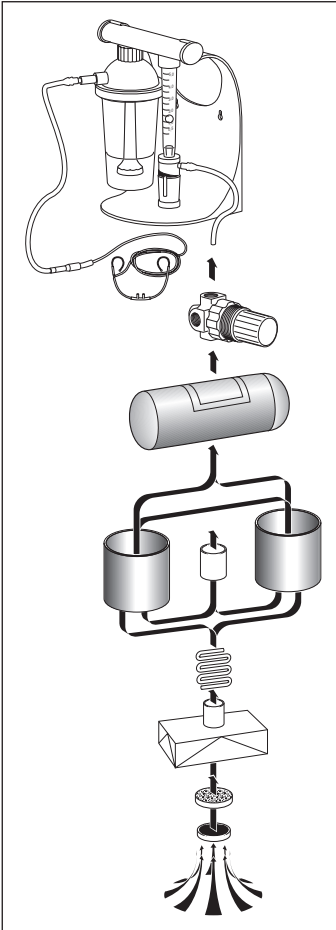
2.2 Function

This section describes the main technical aspects of how the OXYMAT 3 works.

- An electrically powered compressor draws in ambient air through a dust filter and fine-mesh filter and forces the slightly compressed air into containers fitted with molecular filters.
- The molecular filters have the property of absorbing the nitrogen contained in the air.
- Oxygen-enriched air flows from the containers into the oxygen tank. It then passes through a pressure regulator to the oxygen outlet of the OXYMAT 3 oxygen concentrator.
- The desired volume (flow) of oxygen can be set with the oxygen control knob **24**. If you are using a flowmeter, use knob **27** to control the flow. You can read off the flow rate at the top of the flowmeter ball **22**.
- Each time the device is switched on, the control system performs a self-test of the entire O₂ concentrator. This test lasts approximately a minute. If, during this self-test, a fault is found in the O₂ concentrator which leads to O₂ concentration dropping below 82 %, yellow status indicator **19** is switched on.
- After the self-test, the OXYMAT 3 resumes normal mode. The process control system continuously monitors the electrical and the pneumatic circuits of the oxygen concentrator.

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3. Safety Information

For your own safety please observe the following points:

- Read these operating instructions carefully. They are part of the OXYMAT 3 and must be kept available at all times.
- Also be sure to observe the operating instructions for all accessories.
- Use the OXYMAT 3 for the designated purpose only.
- Your OXYMAT 3 oxygen concentrator enriches the ambient air with oxygen. During therapy with the OXYMAT 3 your clothing may come into contact with oxygen-enriched air. This makes the clothing more flammable. For this reason smoking, naked lights (candles) or open fires (fireplace) are prohibited in the immediate vicinity.

This also applies after therapy until the increased oxygen concentration in the clothes has returned to normal.

Please keep in mind: It is for your own safety!

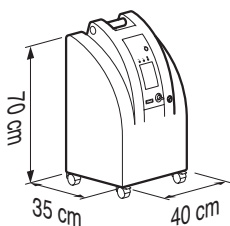
- The mains voltage of the device must correspond to your mains voltage at home.
- During operation, your OXYMAT 3 draws in ambient air. For this reason the devices must not be covered up. Make sure that the air intake slits are always kept free.
- Make sure that the air drawn in does not contain any smoke, vehicle exhaust gases or impurities.

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Español

Oxygen appliance!
Increased fire risk!

- No smoking, no fire or naked lights!
- Keep appliance and flexible tubes free from oil and grease!
- Do not remove any covers from appliance!
- Observe instructions for use!

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- The air drawn in by the concentrator is also used for cooling. Do not place the OXYMAT 3 in front of a heater or in direct sunlight.
- Do not place the OXYMAT 3 in damp rooms (e.g. bathroom) or on the balcony. Persistent absorption of moisture damages the molecular filters and can lead to failure of the OXYMAT 3.
- Remember to clean the coarse dust filter **13** once a week.
- Follow the section entitled "Hygienic preparation" to avoid infection or contamination with bacteria.
- If red fault indicator **18** comes on and an acoustic alarm sounds at the same time, switch off the OXYMAT 3 and contact your authorized dealer.
- If yellow status indicator **19** comes on, together with green power indicator **20**, you can continue to use the device. The indicator gives early warning of an internal fault in the device which should be remedied by your authorized dealer.
- Incorrect use or dosage of oxygen can lead to side effects. It is therefore advisable to seek medical advice before using the equipment.
- Do not use mobile telephones in the immediate vicinity of the OXYMAT 3.
- In view of the fire risk, do not use any lubricants (grease, oil, alcohol) on the OXYMAT 3.
- After long periods of non-use (several months), have the oxygen concentration of the OXYMAT 3 checked by an authorized dealer.
- Only products recommended in these operating instructions should be used with the OXYMAT 3. Proper functioning

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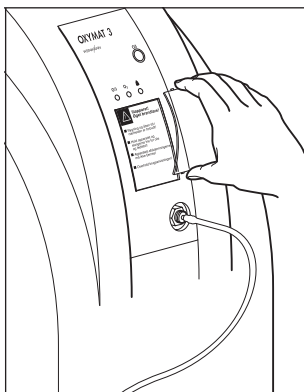
cannot be guaranteed if it is used with other products.

- Malfunctions and a lack of biocompatibility may result if third-party articles are used. Please bear in mind that in these cases any warranty entitlement and liability shall lapse where the accessories recommended in the instructions for use or original spare parts are not utilised.
- When using sterile water systems, use a 500 ml Kendall Respiflo with a UNF 9/16" connecting thread.

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4. Installation

4.1 Setting up device

Fix the enclosed card with warning notices in your own language to the OXYMAT 3.

Please observe the following when choosing the site for the device:

- Choose a well-ventilated position where the OXYMAT 3 does not cause problems.
- This site should be somewhere central in your home, so that you can move around as freely as possible with the dosage monitor **9** and the long 20-meter connecting tube **7**.
- Place the OXYMAT 3 in another room if you are disturbed by the noise it makes.
- Your OXYMAT 3 should stand in a level and dry place. Do not put the OXYMAT 3 in a damp room or on the balcony.
- The air drawn in by the concentrator is also used for cooling. Do not place the OXYMAT 3 in front of a heater or in direct sunlight.
- You are recommended to put it on a smooth floor to ensure unimpeded air circulation. Avoid placing the device on thick carpets.

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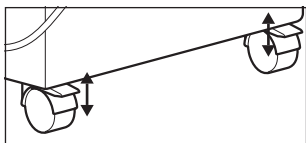
Arresting and releasing the OXYMAT 3 castors

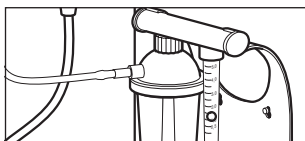
OXYMAT 3 has a brake on each of the front castors.

- Press the brake lever down with your foot to arrest the castors.
- To release the brake, lift the brake lever.

The long hose is connected to the oxygen outlet of the OXYMAT 3.

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4.2 Connecting to dosage monitor and nasal cannula

1. Screw connecting nipple with union nut **6** onto oxygen outlet **5**.
2. Attach one end of long 20-meter connecting tube **7** to oxygen outlet **5** of the OXYMAT 3 and the other end to connection **23** on the dosage unit of the dosage monitor **9**.
3. Now attach short connecting tube **11** (3 meters) to nasal cannula **12** and to connection on bubble humidifier **26**.
4. Now you can use the power cord to connect the OXYMAT 3 to a power outlet. Please note that it needs a mains voltage of 230 V. Please make sure that all connections are firm.

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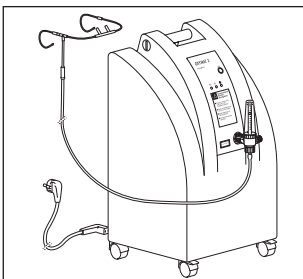
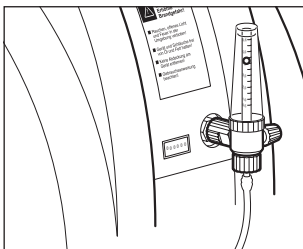
4.3 Connecting to flowmeter and nasal cannula

1. Screw connection **28** of flowmeter **16** to oxygen outlet **5** of the OXYMAT 3.
2. Now attach the 10-meter connecting hose **17** to nasal cannula **12** and to connection **28** on the flowmeter.
3. Now you can use the power cord to connect the OXYMAT 3 to a power outlet. Note that it needs a mains voltage of 230 V.

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
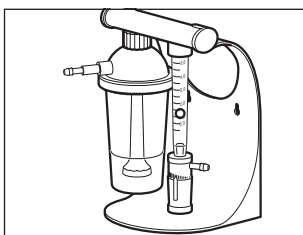
5. Operation

5.1 Before switching on

- If the device has been stored in a cold environment (below +10 °C) before being switched on, put it in a warm room and wait approximately 30 minutes before switching it on to allow the OXYMAT 3 to come to room temperature.
- Unscrew humidifier glass **25** of bubble humidifier from dosage monitor **9** and fill glass with distilled (aqua dest. from your pharmacy), demineralised or boiled water up to the mark. Then screw humidifier glass on again.
- If you use a sterile water system, fit this in accordance with the appropriate installation instructions.

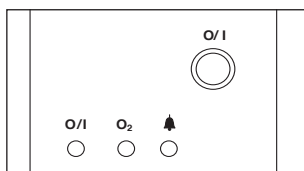5.2 Switching on and off

1. Switch on the OXYMAT 3 using On/Off knob **2**. A self-test will then run.
 - Red fault indicator **18**, yellow status indicator **19** and green power indicator **20** immediately come on briefly.
 - Green power indicator **20** then flashes for the duration of the self-test (approx. 1 minute).
 - During the self-test you will hear the compressor stopping and system pressure being released after approx. 30 seconds (hissing noise). This is an internal leak test taking place. The compressor then starts up again.
 - If the device is functioning perfectly, only green power indicator **20** will still be on after the self-test.

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
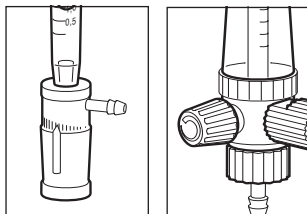
Español

- Yellow status indicator **19** gives warning of a fault. You should then have the device checked by your authorized dealer.
- Red fault indicator **18** indicates a severe fault. The OXYMAT 3 should be taken out of service.
- Please bear in mind that maximum concentration levels are not reached until the device has been in use for about 10 minutes.

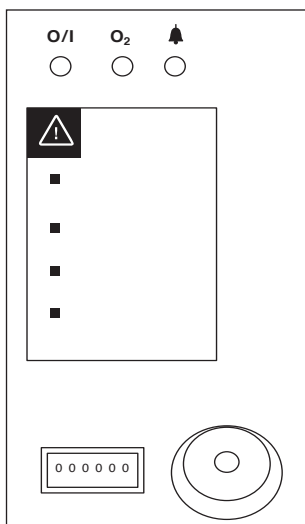2. Use control knob **24** on dosage unit **21** to set the flow prescribed by your doctor. Turn knob **27** on flowmeter **16** to set the flow. Read off flow at top of ball **22**. The scale is marked in liters per minute.
3. The OXYMAT 3 is ready to use. You can now put on the nasal cannula **12**.
4. After treatment, switch off the OXYMAT 3 by pressing On/Off knob **2**. The flow setting can stay in the selected position.

5.3 Indicators

Indicators on the OXYMAT 3

The operating hours meter **4** allows you to keep a check on servicing intervals.

- Green power indicator **20**:
 - always comes on when the OXYMAT 3 is switched on
 - flashes during the self-test after the device is switched on

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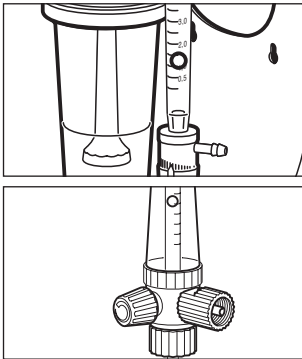
- Yellow status indicator **19** of the O₂ concentrator; only comes on if a fault is detected during the self-test which might lead to reduced performance of the device. In this case, contact your authorized dealer.
- Red fault indicator **18**; this lamp comes on in the event of either a power failure or the device failing. You will simultaneously hear an intermittent alarm (see "8. Troubleshooting" page 110).

Indicator on dosage unit

You can read off the flow rate at the top of ball **22** in the dosage unit. The flow is indicated in liters per minute.

Indicator on flowmeter

You can read off the flow rate at the top of ball **22** on the flowmeter **16**. The flow rate is indicated in liters per minute.

5.4 Storing OXYMAT 3

If you will not be using the device for some time, be sure to clean and dry the humidifier, tube system and nasal cannula before storing them.

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5.5 Bringing the OXYMAT 3 back into service

If the OXYMAT 3 has been out of service for over 6 months, a functional check (see "7. Functional Check" page 106) and an oxygen concentration measurement must be performed by an authorized dealer before the device is brought back into service.

5.6 Tips

- Weinmann supply a range of portable oxygen therapy devices as a reserve for home use, e.g. in the event of power failure.
- For those who need an oxygen supply when on the move, Weinmann offer the OXYTRON 3 Mobil, which enables you to be away from home for a considerable period of time without having to interrupt your oxygen therapy. Ask your doctor or authorized dealer about this possibility.
- The 20-meter connecting hose **7** between the OXYMAT 3 and the dosage monitor **9** makes for great freedom of movement within the home. The dosage monitor can be attached directly to the OXYMAT 3, but it is more convenient to have it with you wherever you are. This means you only need to carry the dosage monitor to where you want it, leaving the OXYMAT 3 standing in one place all the time.

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No water or disinfectant must be allowed to enter the air slits or the tubes.

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6. Hygienic preparation

The OXYMAT 3 and its accessories must always be subjected to hygienic preparation at regular intervals. After performing hygienic preparation please always carry out a functional check (see "7. Functional Check" page 106).

6.1 Time periods

Cleaning of the device and accessories must be performed at the intervals given below.

We recommend also carrying out disinfection within these periods.

- **After every usage:**
 - nasal cannula
 - connecting tubes
- **Daily:**
 - sterile water tank
- **Weekly:**
 - dosage monitor
 - flowmeter
 - coarse dust filter
 - OXYMAT 3 housing
 - power cord

In the case of patients with a weakened immune system or a special history of illness disinfection of the individual components may be necessary after every usage. Please consult your doctor in this case.

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6.2 Preparation

When using a dosage monitor

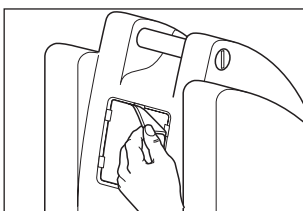
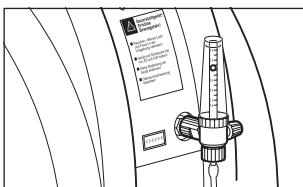
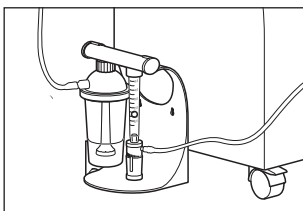
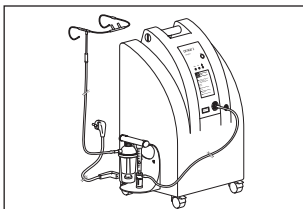
1. Remove the short connecting tube (3 m) **11** from the nasal cannula **12** and from the connection of the bubble humidifier **26**.
2. Remove the long connecting tube (20 m) **7** from the oxygen connection **23** of the dosage monitor and from the oxygen outlet **5** of the OXYMAT 3.
3. Unscrew the sterile water tank from the dosage monitor. For further preparation of the sterile water tank please observe the relevant operating instructions.

When using a flowmeter

1. Remove the long connecting tube **17** (10 m) from the connection **29** of the flowmeter.
2. Release the union nut **28** of the flowmeter and disconnect the flowmeter from the oxygen outlet of the OXYMAT 3.

Removal of coarse dust filter

Please detach the coarse dust filter **13** from the service flap at the rear of the OXYMAT 3.

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6.3 Cleaning and disinfection

Carry out hygienic preparation of the OXYMAT 3 and accessories as described in the table given below.

For cleaning we recommend using a mild household detergent. For wipe disinfection we recommend TERRALIN. Please observe the directions for use of the disinfectant used.

Note!

When carrying out disinfection we recommend wearing suitable gloves (e.g. household or disposable gloves).

Components	Cleaning	Disinfection
Nasal cannula	External under running water	Wipe disinfection
Connecting tubes	External under running water	Wipe disinfection
Sterile water tank	See operating instructions of sterile water tank	
Dosage monitor	Wipe with damp cloth	Wipe disinfection
Flowmeter	Wipe with damp cloth	Wipe disinfection
Coarse dust filter	Dismantle and brush off / vacuum	
OXYMAT 3 housing	Wipe with damp cloth	Wipe disinfection
Power cord	Wipe with damp cloth	Wipe disinfection

Caution!

Make sure that water or disinfectant does not get into air intake slits or tubes!

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6.4 Sterilisation of sterile water tank

The sterile water tank can also be steam-sterilised or autoclaved. This is generally only necessary when the device is to be used by more than one patients (change in patients).

Detailed information about sterilisation can be found in the operating instructions of the sterile water tank.

6.5 Assembly

To assemble the system proceed in reverse order as described under "4. Installation" on page 84.

6.6 Change in patients

If the unit is to be utilized for another patient, it must undergo a hygiene treatment beforehand. This has to be carried out by the manufacturer Weinmann or a dealer.

The procedure used for the hygiene treatment is described in the inspection sheet and the service and repair instructions for OXYMAT 3.

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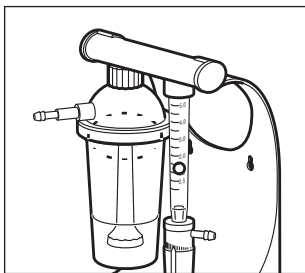
7. Functional Check

1. Switch on the OXYMAT 3 using On/Off knob **2**. A self-test will then run.
 - Red fault indicator **18**, yellow status indicator **19** and green power indicator **20** immediately come on briefly.
 - Green power indicator **20** then flashes for the duration of the self-test (approx. 1 minute).
 - If the device is functioning perfectly, only green power indicator **20** will still be on after the self-test.
 - Yellow status indicator **19** gives warning of a fault. You should then have the device checked by your authorized dealer.
 - Red fault indicator **18** indicates a severe fault. The OXYMAT 3 should be taken out of service.
2. Check that the red minutes counter on the operating time meter **4** moves on every 6 minutes.
3. Check that the screw connections between the cover and glass of the bubble humidifier and between the humidifier and dosage monitor are firm. If necessary, tighten these connections by hand.
4. Set the flow rate to 2 liters per minute by means of the oxygen dosage control knob **24**.
5. To check the dosage monitor **9** for leaks, bend 3-meter tube **11**, or pull 3-meter hose **11** off the connection on bubble humidifier **25** and hold connection **26** closed. The dosage monitor system is free from leaks if the flowmeter ball **22** sinks all the way to the bottom. In the event of leaks you should replace the sealing rings on the bubble humidifier.

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Español

6. To check flowmeter **16** for leaks, bend the connecting tube **17**. Or pull connecting tube **17** off connection **28** on flowmeter and hold connection **28** closed. The flowmeter is free from leaks if the flowmeter ball **22** sinks all the way to the bottom.
7. Check mains failure alarm. To do this, pull out the mains plug while the OXYMAT 3 is switched on. An acoustic signal is heard, and the red fault indicator **18** lights up.
8. Insert the mains plug in the power socket again. The alarm signal should stop, and the device starts again.
9. Switch off your OXYMAT 3 when you no longer need it.

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8. Troubleshooting

8.1 General

Fault	Cause	Remedy
No sound of motor, power indicator does not light up, mains failure alarm sounds.	Faulty fuse	Change fuse (see "8.2 Changing fuses" page 114).
	No power	Ensure power supply.
Device running, but intermittent a-coustic warnings are heard and red fault indicator flashes.	Device running hot, air intake slits blocked.	Clean air intake slits. Allow device to cool down.
After running for a long period, device emits intermittent acoustic warnings and red fault indicator flashes, no sound of motor to be noticed.	Compressor not running.	Check whether air intake slits are blocked. Allow device to cool thoroughly. Restart device after a while. If the warning signal sounds again, the device is defective. Consult authorized dealer.
Flow at outlet from device, but not at nasal cannula. No warning from device.	Faulty tube connections. Bend tubes. Faulty seals or leak in screw connection	First check whether flow is indicated at the dosage monitor. Check all tube connections and screw connections for leaks, and replace seals if necessary.
Inadequate flow	Oxygen distributor in bubble humidifier blocked.	Clean oxygen distributor or replace if necessary.

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Fault	Cause	Remedy
Yellow status indicator for O ₂ concentrator comes on.	A fault has been found in the O ₂ concentrator which, in the long term, may lead to O ₂ concentration dropping below 82 %.	Please contact your authorized dealer.
Power cord damaged		Replace power cord.

If you cannot rectify a fault immediately, you should contact your authorized dealer or the manufacturer. Send faulty devices to your authorized dealer or to the manufacturer (Weinmann, Siebenstücken 14, D-24558 Henstedt-Ulzburg, Germany).

If the repairs are likely to take a long time, you will be given a new device to continue your therapy (OXYMAT 3 or O₂ bottle unit).

Never use OXYMAT 3 if the red fault indicator is on.

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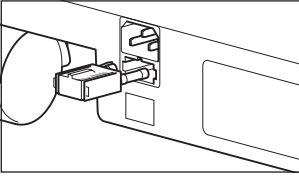
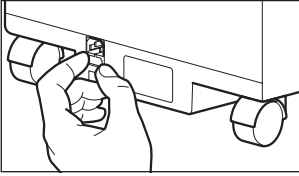
8.2 Changing fuses

At the rear of the OXYMAT 3 are two fuses of type T 2.5H / 250V.

1. First switch off the device, then pull out the mains plug.
2. Squeeze together the tabs on each side of the fuse holder **15** and pull the holder out.
3. Change the faulty fuse. You will find a spare fuse in the fuse holder.
4. Push the holder with the new fuse back into the device.

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Only the original carton should be used for shipping the device. We therefore recommend you to keep the carton in a safe place.

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9. Maintenance

Note:

Please remember that a final check should be made after every start-up.

We recommend preventive servicing of the OXYMAT 3 oxygen concentrator by a trained authorized dealer or by Weinmann, the manufacturers:

- after 5000 operating hours (see operating hours meter). For safety, check whether the 5000 operating hours have been reached.
- when a year has elapsed since the last service.
- when the device has not been in service for over six months.

Servicing should include:

- changing the coarse dust and suction filter
- cleaning the device
- checking the device equipment for completeness
- a visual inspection for:
 - mechanical damage, in particular to the power cord
 - legible labelling of controls
 - damage to any tube connections
- final check according to test record WM 13004 or servicing and repair instructions.

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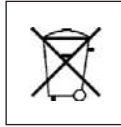
Attention:

The flow behaviour of the suction filter is not impaired whenever it has turned dark in colour as long as the number of operating hours is still below 5,000 h, unless the device is used in a very dusty environment.

For hygiene reasons, do not send the tube systems or nasal cannula with the device.

10. Disposal

Do not dispose of the unit with domestic waste. To ensure proper disposal of the device, consult an authorized waste disposal company.

You can find out their address from your environmental officer or municipal cleaning department.

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11. Product and Spare Parts

11.1 Standard products

The OXYMAT 3 can be ordered in six different variants as standard.

Variants	Order No.
– OXYMAT 3 with separate dosage monitor, for adults	WM 13011
– OXYMAT 3 with separate dosage monitor, for children	WM 13021
– OXYMAT 3 with flowmeter, for adults	WM 13031
– OXYMAT 3 with flowmeter, for children	WM 13041
– OXYMAT 3 with dosage monitor for sterile water systems for adults	WM 13017
– OXYMAT 3 with dosage monitor for sterile water systems for children	WM 13018

WM 13011: OXYMAT 3 with separate dosage monitor, for adults, comprising

Parts	Order No.
– OXYMAT 3 oxygen concentrator	WM 13005
– Dosage monitor for adults 0,5 - 5l/min	WM 13700
– Connecting tube, 3 m	WM 5720
– Connecting tube, 20 m	WM 12330
– Nasal cannula, with earpieces	WM 1929

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WM 13021: OXYMAT 3 with separate dosage monitor, for children, comprising

Parts	Order No.
- OXYMAT 3 oxygen concentrator	WM 13005
- Dosage monitor for children 0,2 - 2l/min	WM 13770
- Connecting tube, 3 m	WM 5720
- Connecting tube, 20 m	WM 12330
- Nasal cannula, baby	WM 1920
- Nasal cannula, child	WM 1921
- Power Cord	WM 13421
- Operating instructions OXYMAT 3	WM 16265

WM 13031: OXYMAT 3 with flowmeter, for adults, comprising

Parts	Order No.
- OXYMAT 3 oxygen concentrator	WM 13005
- Flowmeter, adjustable, 0,5 - 5 l/min	WM 13650
- Connecting tube, 10 m	WM 13015
- Nasal cannula, with earpieces	WM 1929
- Power Cord	WM 13421
- Operating instructions OXYMAT 3	WM 16265

WM 13041: OXYMAT 3 with flowmeter, for children, comprising

Parts	Order No.
- OXYMAT 3 oxygen concentrator	WM 13005
- Flowmeter, adjustable, 0,2 - 3 l/min	WM 13670
- Connecting tube, 10 m	WM 13015
- Nasal cannula, baby	WM 1920
- Nasal cannula, child	WM 1921
- Power Cord	WM 13421
- Operating instructions OXYMAT 3	WM 16265

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Svenska

Español

WM 13017: OXYMAT 3 with dosage monitor for sterile water systems for adults, consisting of

Parts	Order No.
– OXYMAT 3 oxygen concentrator	WM 13005
– Dosage monitor for sterile water systems for adults, 0.5 - 5 l/min.	WM 13715
– Connecting tube, 3 m	WM 5720
– Connecting tube, 20 m	WM 12330
– Oxygen mask with ear loops	WM 1929

WM 13018: OXYMAT 3 with dosage monitor for sterile water systems for children, consisting of

Parts	Order No.
– OXYMAT 3 oxygen concentrator	WM 13005
– Dosage monitor for sterile water systems for children, 0.2 - 3 l/min.	WM 13716
– Connecting tube, 3 m	WM 5720
– Connecting tube, 20 m	WM 12330
– Oxygen mask, baby	WM 1920
– Oxygen mask, child	WM 1921

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11.2 Spare parts

Parts	Order No.
Connecting tube, 3 m	WM 5720
Connecting tube, 10 m	WM 13015
Connecting tube, 20 m	WM 12330
Nasal cannula, baby	WM 1920
Nasal cannula, child	WM 1921
Nasal cannula, with earpieces	WM 1929
Fuse insert DIN EN 60127-2 (T 2.5 H 250 V)	WM 13422
Dosage monitor for adults	WM 13700
Dosage monitor for sterile water systems for adults	WM 13715
Dosage monitor for children	WM 13770
Dosage monitor for sterile water systems for children	WM 13716
Flowmeter, adjustable, 5 L/min	WM 13650
Flowmeter, adjustable, 3 L/min	WM 13670
Bubble humidifier	WM 13730
Sealing ring for connection nut	WM 1145/31
Sealing ring, bubble humidifier	WM 1145/93
Coarse dust filter set	WM 15360
Connecting nipple with union nut	WM 1470
Operating instructions OXYMAT 3	WM 16641
Operating instruction, bubble humidifier	WM 16199
Operating instructions Flowmeter	WM 16198

Subject to technical change without notice.

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11.3 Accessories

Parts	Order No.
Adapter for sterile water systems	WM 13755

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Español

12. Technical Data

	OXYMAT 3
Product category according to 93/42/EEC	Ila
Dimensions WxHxD in mm	400 x 700 x 350
Weight	approx. 20 kg
Temperature range – Operation – Storage	+10 to +40 °C - 20 to +70 °C
Power supply	230V, 50 Hz
Power output / current consumption	360 W / 1.6 A
Mains fuse DIN EN 60127-2	T 2.5 A H 250V
Classification acc. to EN 60601-1 – Protection against electric shock – Degree of protection against electric shock	Class 2 Type B
Electromagnetic compatibility – Radio interference suppression – Radio interference resistance	EN 55011 (VDE 0875 Pt11) IEC 1000-4 Parts 2-6 and 11
Sound pressure level	≤ 40 dB(A) (from serial number 20.000)
O ₂ output (values after 10 min. operation at + 20 °C, 50% relative humidity and 1013 mbar)	95 ± 1/3 vol% O ₂ at 1–4 l/min 90 ± 3 vol% O ₂ at 5 l/min
O ₂ output at 2000 m above msl	90 ± 3 vol% O ₂ at 5 l/min
Oxygen outlet pressure	60 kPa
Flow rate (read at top of ball)	0.5 – 5.5 l/min 0.2 – 3.0 l/min
OSCI O ₂ status indicator as per DIN EN ISO 8359.51.5 Warning at < 82 % by vol. of O ₂	present
Fault warning	Visual and acoustic
Actuating pressure of pressure relief mechanism	300 kPa
Servicing	at least once a year or every 5000 operating hours

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Español

13. Warranty

- Weinmann offers a warranty that the product, when used in accordance with requirements, will remain free from defects for a period of five years from date of purchase. For products whose durability is clearly indicated as less than five years, the warranty expires on the expiration date indicated on the packaging or in the user's manual.
- Claims against the warranty can be made only when accompanied by the sales receipt, which must show salesperson and date of purchase.
- We offer no warranty in the case of:
 - Disregard of usage instructions
 - Operating errors
 - Improper use or handling
 - Third-party intervention by non-authorized persons for the purpose of device repair
 - Acts of God, e.g., lightning strikes, etc.
 - Transport damage as a result of improper packaging of returned items
 - Lack of maintenance
 - Operational and normal wear and tear, which includes, for example, the following components:
 - Filter
 - Batteries and rechargeable batteries
 - Articles for one-time usage, etc.
 - failure to use original spare parts.

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Español

- Weinmann is not liable for consequential harm caused by a defect if it is not based on intention or gross negligence. Weinmann is also not liable for minor physical injury to life or limb resulting from negligence.
- Weinmann reserves the right to decide whether to eliminate defects, to deliver a defect-free item or to reduce the purchase price by a reasonable amount.
- If Weinmann rejects a claim against the warranty, it assumes no expense for transport between customer and manufacturer.
- Implied warranty claims remain unaffected by these changes.
- Weinmann additionally provides a three-year warranty on the components of the compressor and printed circuit (electronics). This only includes replacement of materials after expiry of the first 2 years.

EEC Letter of Conformity on Medical Products	
We, the manufacturer	GOETTER WEINMANN Geräte für Medizin und Arbeitsschutz GmbH+Co. Königsallee 40 D-22525 Hamburg
declare in sole responsibility the	Product Name: Oxygen Concentrator Type/Model: OXYMAT 3
being in conformity with the respective regulations of the following guideline:	Directive 93/42/EEC on Medical Products
Classification:	I a
Standards used in particular:	EN 60601-1: 1990 + A1: 1993 + A2: 1995 EN 60601-1-2: 1993 DIN EN ISO 1359: 1996
Hamburg, 27.01.1998	
 Dr. J. K. A. Fiedhorn Executive President Chief Executive Officer	 Dr. K. A. Fiedhorn Executive Vice President Chief Operating Officer
WEINMANN	

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Español

For decades Weinmann has been developing, producing and marketing medical devices for markets around the world. In cooperation with our partners we design economic health systems for diagnosis and therapy in Sleep Medicine, Home Mechanical Ventilation, Oxygen Medicine and Emergency Medicine.

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Geräte für Medizin GmbH+Co. KG